
VOCO

Rebilda® Post Drills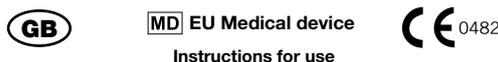

Instructions for use

Rebilda Post drills are available in four colours corresponding to the root posts (colour-coded).

Size	10 (red)	12 (green)	15 (black)	20 (yellow)
Diameter	1.00 mm	1.26 mm	1.56 mm	2.06 mm

Indications:

– Preparation of the post bed in the scope of postendodontic treatment.

Contraindications:

– Circular coronal tooth tissue measuring less than 1.5 mm.

Patient target group:

Rebilda Post drills is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Rebilda Post drills should only be used by a professionally trained dental practitioner.

Use:

A.) Precision drilling

The prerequisite is a successfully completed endodontic treatment. The corresponding post size should be selected on the basis of an x-ray image and the corresponding working length determined (at least 4 mm of root-canal filling must be preserved apically; the post should be located approximately 2/3 in the root canal and 1/3 in the core build-up).

The root-canal filling is removed down to the planned depth with a Gates-Glidden drill or Peeso reamer, followed by precision drilling with the corresponding bur (1,000 – 2,000 rpm, green contra-angle). Avoid reprocessing beyond the apex and lateral perforation of the root. [The burs and Rebilda Post root posts must always be used together as a system (compare colour-coding) and must not be used in combination with other brands]. In order to guarantee optimal drilling capacity, the bur and canal should be rinsed intermittently to remove remnants of dentine.

B.) Revision

- X-ray images for locating the post and determining its length and size
- Remove the restoration and core build-up so as to allow direct, straight access to the post
- Perform initial drilling in the centre of the post with a rose head bur
- X-ray image to check direction of drilling if required
- Excavate Rebilda Post root posts with the corresponding bur (Caution: The similar cutting behaviour of the rotary instruments in the glass-fibre-reinforced root post and root dentine means the practitioner must work with extreme care in order to avoid perforating the root. If necessary, check the success of the intermediate steps using x-ray images).

Cleaning, disinfection, sterilisation (burs are not sterile when delivered):

- Perform a visual inspection of the bur, clean it and autoclave it (134°C, 18 minutes) before every use. Use a separate autoclave pouch for each instrument. Reclean any burs which are not completely clean. Sort out any damaged or blunt burs and do not use any further.
- Use agents which are suitable for the cleaning and disinfection of rotary instruments (in accordance with EN 1040 and EN 1275). The respective manufacturer's specification must be observed and followed.

Some aldehyde-free cleaning and disinfectant solutions can result in discoloration of the colour-coded rings. Use a cleaning brush or ultrasound bath (position items such that they can't come in contact with each other) for the cleaning. Then rinse thoroughly with water and dry. Check the complete success of the cleaning and repeat if necessary.

Warnings, precautionary measures:

– Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Store the Rebilda Post drills in a dry, dust-free location at between 4°C and 23°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Rebilda Post drills must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA1777 E1 0720 99 © by VOCO