VOCO

Ceramic Bond

CE 0482

EN Instructions for use
MD EU Medical device

Product description:

Ceramic Bond is a one-component silanisation agent for durable adhesive bonding between indirect restorations (also zirconium dioxide) and luting composites, as well as for the pretreatment of glass fibre reinforced composite posts.

Indications:

Ceramic Bond is used as a coupling agent to achieve an optimal bond strength between indirect restorations made of either silicate ceramics, aluminium oxide, zirconium dioxide, metal or composite, as well as glass fibre-reinforced composite posts, on the one side and methacrylate-based luting composites on the other side.

Contraindications:

Ceramic Bond contains methacrylated phosphoric acid ester and 3-methacryloxypropyltrimethoxysilane. **Ceramic Bond** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Ceramic Bond is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Ceramic Bond should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Bring material to room temperature before use.

Luting of indirect restorations:

Surface conditioning of indirect restorations, including those made from zirconium dioxide, creates optimum conditions for longterm bonding of restoration and luting composite.

1. Once the fit of the restoration has been checked, prepare its bonding surfaces according to the manufacturer's instructions for use by etching or sandblasting, rinse carefully with water and dry with oil-free air.

2a. When using the bottle:

Dispense **Ceramic Bond** onto a **Single Tim** or a disposable brush. Alternatively, dispense **Ceramic Bond** onto a mixing pallet. Close the bottle after use.

2b. When using the *SingleDose*:

Detach a **SingleDose** at the perforated line and turn the printed side upwards. Pierce the foil with a **Single Tim**, enlarge the opening and moisten the brush with liquid through circular motion.

3. Apply **Ceramic Bond** onto the prepared surfaces with a **Single Tim** or disposable brush, allow to take effect for 60 s and dry carefully with oil-free air.

4. Apply the luting composite as described in the respective instructions for use and place the restoration.

Luting of glass fibre reinforced posts:

1. After appropriate preparation of the root canal and position check of the glass fibre reinforced post, shorten the post to operational length according to the manufacturer's instructions for use, then clean with alcohol and dry with oil-free air.

2. Coat the entire post with **Ceramic Bond** (for details on dispensing from bottle or **SingleDose**: see above), allow to take effect for 60 s and dry carefully with oil-free air.

3. Cement the post in the root canal according to the instructions of the manufacturer.

Warnings, precautionary measures:

- Contamination of the conditioned bonding surface (e.g. saliva or blood) may reduce the bonding strength of **Ceramic Bond**. Therefore, repeat the surface conditioning steps as described above.
- Contact of **Ceramic Bond** with skin/mucosa or eyes may have an irritating effect and should be avoided.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Acetone, 3-methacryloxypropyltrimethoxysilane, methacrylated phosphoric acid ester

Storage:

Storage at 4°C-23°C. Refrigeration is recommended. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Ceramic Bond** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Ceramic Bond** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 BB1107 E1 0225 99 © by VOCO