Monobond® Etch & Prime

[en] Instructions for Use

Ceramic Primer

Rx ONLY

C € 0123

Date information prepared:
2024-02-26 / Rev. 0

Manufacturer:
Ivoclar Vivadent AG
Bendererstrasse 2
9494 Schaan / Liechtenstein
www.ivoclar.com

ivoclar

English

1 Intended use

Intended purpose

Preparing indirect dental restorations made of silicate ceramics for adhesive cementation.

Patient target group

- Patients with permanent teeth
- Adult patients with dental implants

Intended users

- Dentists
- Dental assistants
- Dental technicians

Special training

No further special training is needed.

Use

For dental use only.

Description

Monobond® Etch & Prime is a self-etching single component ceramic primer for pre-treated and non pre-treated glass-ceramic materials. Monobond Etch & Prime is used to condition silica-based ceramic surfaces for the adhesive bond with luting composites. Monobond Etch & Prime etches and silanizes silicate ceramic surfaces in one working step.

Indications

- Missing tooth structure in anterior and posterior teeth
- Partial edentulism in the anterior and posterior region

Areas of application/Types of restoration:

Monobond Etch & Prime serves as a bonding agent and is used to create a durable chemical bond between luting composites and glass-ceramic restorations.

Contraindications

The use of Monobond Etch & Prime is contraindicated if the patient is known to be allergic to any of its ingredients.

Limitations of use

- Intraoral application
- Conditioning of oxide ceramic restorations, metal restorations and composite restorations.
- To avoid any contamination, Monobond Etch & Prime must not be removed directly from the bottle using an applicator.
- Do not insert the applicator into the Monobond Etch & Prime bottle. This may lead to contamination.
- Carefully close the Monobond Etch & Prime bottle immediately after use to prevent evaporation of the solvent.

Side effects

In rare cases, components of Monobond Etch & Prime may lead to sensitization. The product should no longer be used in such cases.

Interactions

There are no known interactions to date.

Clinical benefit

- Reconstruction of chewing function
- Restoration of esthetics

Composition

Water, alcohol, ammonium-polyfluoride, phosphoric acid methacrylate, silane methacrylate, dipodal silane, colouring agent.

2 Application

Preparation of pre-treated and non pre-treated silicate ceramic restorations

- Following try-in, thoroughly rinse the restoration with water spray and dry with oil- and moisture-free air.
- Apply Monobond Etch & Prime onto the bonding surface using a microbrush and agitate it into the surface for 20 seconds using light pressure. Allow to react for another 40 seconds.
- Thoroughly rinse off Monobond Etch & Prime with water until the green colour has been removed. If any residue remains in micro-porosities after rinsing, the restoration may be cleaned with water in an ultrasonic unit for up to 5 minutes.
- Dry the restoration with a strong stream of oil- and moisture-free air for approx. 10 seconds.

Note: Contamination of surfaces conditioned with Monobond Etch & Prime with saliva and blood must be prevented.

Insert the conditioned restoration using a luting composite in accordance with the respective Instructions for Use.

3 Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclar.com and your responsible competent authority.
- The current Instructions for Use and explanation of the symbols are available on the website: www.ivoclar.com/eIFU.
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.
- Basic UDI-DI: 76152082ABOND008HY

Warnings

- Observe the Safety Data Sheet (SDS) (available at www.ivoclar.com).
- The product contains alcohols. Keep away from sources of ignition.
- Monobond Etch & Prime is corrosive. Avoid contact with the skin and mucous membrane. If the material should accidentally come in contact with the skin, immediately wash with copious amounts of water.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Disposal information

Remaining stocks must be disposed of according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks.

The following clinical residual risks are known:

- Failure of the adhesive bond

4 Shelf life and storage

- Storage temperature 2–28 °C (36–82 °F)
- Do not use the product after the indicated date of expiration.
- Expiry date: see information on bottle and/or package.
- Before use, visually inspect the packaging and the product for damage. In case of any doubts, please contact Ivoclar Vivadent AG or your local dealer.

5 Additional information

Keep material out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions.

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- Jei kyla abejonių, kreipkitės į „Ivoclar Vivadent AG“ arba savo vietinį platintoją.