Description

Total Etch is a gel used for enamel etching and dentin conditioning in conjunction with sealing and direct composite restorations as well as adhesive cementation of lab-fabricated all-ceramic or composite restorations.

Composition

Total Etch contains phosphoric acid (37 wt.% in water), thickening agent and colour pigments.

Indications

Total Etch is applied using the enamel etch or total etch technique for the following indications:

- Composite restorations
- Adhesive cementation of inlays (composite, all-ceramic), crowns, bridges, veneers, adhesive bridges, root canal posts
- Splints
- Bonding of orthodontic attachments (e.g. brackets)
- Sealings of fissures

Contraindication

If the stipulated application technique cannot be applied.

Side effects

None known to date.

Interactions

Glass ionomer-based liners may be dissolved by Total Etch.

Application

- Apply Total Etch on enamel and subsequently on dentin; and allow a reaction time of 15 seconds.
- Afterwards, thoroughly rinse off the etchant with water spray and dry the tooth surfaces with oil-free air. Avoid excessive drying of the dentin.
- The etched enamel surface should have a chalky white appearance. If this is not the case, repeat etching with a shorter reaction time.
- If the tooth surfaces becomes accidentally contaminated (e.g. with saliva), repeat the etching procedure with a reaction time of max. 10 seconds.
- The reaction time on unprepared enamel (e.g. with fissure sealants) is 30-60 seconds. If the enamel is selectively etched, Total-Etch has to be left to react for 15-30 seconds.

Refilling empty Total Etch application syringes

Empty 2-ml syringes can be conveniently refilled with the contents of the Total Etch jumbo syringe up to 15 times.

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Important

Although the 2-ml syringes are equipped with a plunger safety stop, it is absolutely necessary to wear gloves, protective goggles and a face mask for safety reasons.

- Remove the screw-on cap from the 30-gram jumbo syringe and the empty 2-ml syringe.
- Join the tips of the two syringes using a twisting, clockwise motion. Make sure that the two syringes are firmly connected. **Note:** Do not strip the thread.
- Refill the 2-ml syringe by slowly and steadily pulling back the plunger of the small syringe. Simultaneously exert light pressure on the plunger of the jumbo syringe using the other hand to support the refilling process.
- **Avoid overfilling the 2-ml syringe.**
- **Under no circumstances should you exceed the plunger safety stop of the 2-ml syringe due to the risk of acid burn!**
- Do not point the syringes at people during refilling.
- Do not fill etchant from the 2-ml syringe back into the jumbo syringe.
- Disconnect the two syringes using a counter-clockwise twist.
- Replace the screw-on cap on the 30-gram jumbo syringe.
- The 2-ml syringe is now ready for use. Attach a Total Etch application tip onto the 2-ml syringe. Apply the etchant as described above.
- Use only the Total Etch application tips delivered with this package. The use of other application tips may have an adverse effect on the viscosity of the etchant.
- In order to ensure traceability of batches, write the LOT number and date of expiration of the 30-gram jumbo syringe on one of the labels included in the delivery form and stick it on the filled 2-ml syringe.

Warning

When filling the 2-ml syringe, always wear protective goggles and gloves. Phosphoric acid is corrosive. Avoid contact with the eyes, mucous membrane and skin (goggles are recommended for both patients and operators). After contact with the eyes, immediately rinse with water and see a physician/opthalmologist. After skin contact, immediately rinse with soap and water. After contact with sensitive materials (surfaces, clothes), quickly clean with water.

Storage

- Do not use Total Etch after the indicated expiration date.
- Storage temperature: 2–28 °C (36–82 °F). Recommendation: Refrigerate large inventory at 2–8 °C (36–46 °F).
- Date of expiration: see note on syringe/package.

Keep out of the reach of children!

For use in dentistry only.

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

The Material Safety Data Sheet is available online at www.ivoclarvivadent.com

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