Pontiform automix 2 / direct mix

Stand:180116

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GB Instructions

Description:

Bis-acrylic composite for temporary crowns and bridges.

M+W Pontiform automix 2 / direct mix is a multifunctional methyl methacrylate-based two-component material for direct application.

The automatic mixing system ensures a uniform quality of material and processing and bubble-free mixing, and saves a considerable amount of time. M+W Pontiform automix 2 / direct mix temporary restorations are produced in the patient's mouth to match the shape of the original teeth and largely correspond to conventional composite materials with regard to their mechanical and cosmetic properties.

Repairs can be carried out quickly and simply, both extraorally using M+W Pontiform automix 2 / direct mix and in the patient's mouth using a light-curing composite.

Indications

M+W Pontiform automix 2 / direct mix is used to fabricate temporary crowns and bridges.

Handling of double syringe

The double syringe automatically dispenses and mixes the material al, which saves time when applying the lifting material directly into the restoration. Remove the transport cap before using a new double syringe and then attach the mixing tip. Following application, the mixing tip acts as a seal and can be left on the double syringe until the next use. The mixing tip should only be replaced by a new tip when the double syringe is used again. When using for the first time, discard the material initially extruded from the mixing tip (about the size of a pea). There is an optimal mixture after initial extrusion.

Cartridge handling

The material is dispensed and mixed automatically by extruding through a mixing tip. Remove the transport seal before using a new cartridge. Then attach the mixing tip and, if required, a conturation tip. Following use, the mixing tip should be used as a seal and should be left on the cartridge until the next application. The mixing tip should only be replaced with a new one when the cartridge is to be used again. When using for the first time, discard the material initially extruded from the mixing tip (about the size of a pea). There is an optimal mixture after initial extrusion.

Application

1. Prepare a cast.

Take an alginate (e.g. M+W Algicrit) or silicone (e.g. M+W Sioplast A) cast before preparing the tooth base or bridge, or before extraction. Keep the alginate casts at 100% humidity (e.g. Hygrophor) until M+W Pontiform automix 2 / direct mix is applied. Thermofomed films or strip crowns may also be used for shaping.

Cut the interdental tabs from the impression so there is enough material for the fabrication process, thus affording the temporary restoration better stability. If the wall of the temporary restoration is expected to be too thin, cut the impression back further.

If there are gaps in the molar region, cut a groove between adjacent teeth so as to create a web-like connection between the prepared teeth.

2. Working sequence

0 min	Syringe into the impression.
0- 45 s	Insert the impression intraorally.
2-3 min	Remove from the patient's mouth.
6 min	Remove the oxygen inhibition layer with a solvent.
6-7 min	Preparation, contouring and polishing.

Important:

- M+W Pontiform automix 2 / direct mix only starts to flow from the mixing tip after a specific amount of time has elapsed. Applying more pressure will not cause the product to flow more quickly. The flow will stop as soon as pressure is removed.

- Do not forcibly squeeze out any material that has already set in the mixing tip. - Risk of defects and leakage in cartridges or syringes and tips.

Approx. 2-3 minutes after the onset of mixing M+W Pontiform automix 2 / direct mix has a low-resilience consistency and can easily be removed from the patient's mouth with the impression. Monitor the setting process by way of the excess in the mouth, as removal of the temporary restoration is only possible during the resilient phase. Higher temperatures (e.g. in the mouth) accelerate the setting of M+W Pontiform automix 2 / direct mix. It is therefore not possible to check the setting of residual material on the mixing block.

Once the temporary resin restoration has completely cured (6 - 7 min), it can be shaped using rotary tools and instruments. Atmospheric oxygen prevents M+W Pontiform automix 2 / direct mix temporary restorations from setting completely at the surface. This greasy layer can easily be removed using alcohol or another solvent.

3. Fixing the temporary restoration

M+W Pontiform automix 2 / direct mix temporary restorations may be used with conventional temporary fixing cements. Cements containing eugenol may hinder the setting of any fixing composites used as a later stage.

4. Repairing a temporary restoration

M+W Pontiform automix 2 / direct mix temporary restorations are characterised by a high level of mechanical stability. However, should the temporary restoration break, the following action is recommended: a) Should the temporary restoration break shortly after fabrication, close the breakage points using freshly mixed M+W Pontiform automix 2 / direct mix. b) Should an older or worn M+W Pontiform automix 2 / direct mix temporary restoration break, roughen the surface of breakage points and provide with undercuts. Close the prepared breakage points using freshly mixed M+W Pontiform automix 2 / direct mix. The repaired temporary restoration may be placed in warm water (50 °C) for a few minutes to accelerate complete polymerization.

Warning:

Patients who have sensitive teeth may experience increased sensitivity when using M+W Pontiform automix 2 / direct mix. Should any reactions occur, cease use.

Storage instructions:

- Do not store above 25 °C.
- Do not use beyond the expiry date.
- It is important that M+W Pontiform automix 2 / direct mix is used at room temperature.
- Cooled material is more viscous and cures more slowly.

Properties:

Compressive strength 200 MPa
Flexural strength 60 MPa
Diam. tensile strength 31 MPa
Barcol hardness 20
Water uptake 25 µg/m³
Maximum setting temperature 38 °C

Composition

Glass filler materials in a matrix of multifunctional methacrylates; catalyst, stabilizers, additives. Free of methyl methacrylate and peroxides. Filler content: 47 % by weight = 26% by volume. The variation width of the inorganic filler particles is between 0.02 and 2.5 µm)

REF	Contents of Pontiform automix 2 packaging
000 216	1 double cartridge (4:1) with 78 g material Colour: universal A2*, 15 Spar-mixing tips

REF	Contents of Pontiform direct mix packaging
000 290	5 ml syringe A2*, 5 mixing tips
000 291	20 direct mix mixing tips.

* roughly corresponds to VITA colour

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