[en] Instructions for Use

Enamel-Dentin Bonding Agent

Rx ONLY

C € 0123

Date information prepared:
2023-12-01 / Rev. 0

Manufacturer:
Ivoclar Vivadent AG
Bendererstrasse 2
9494 Schaan/Liechtenstein
www.ivoclar.com

ivoclar

English

1 Intended use

Intended purpose

Bonding of dental restorations to dental hard tissue

Patient target group

Patients with permanent teeth

Intended users / Special training

- Dentists
- No special training required.

Use

For dental use only.

Description

Note: The product consists of the three individual components Syntac® Primer, Syntac® Adhesive and Heliobond. All three components must be used in combination and in the prescribed sequence. The three components can be purchased individually.

Syntac is a enamel-dentin adhesive assortment for direct and indirect restorations. It can be used in combination with both light- and self-curing composites. The light-curing bonding agent Heliobond must be used prior to the application of the composite.

Indications

- Missing tooth structure in anterior and posterior teeth
- Partial edentulism in the anterior and posterior region

Areas of application:

- Adhesive for direct composite restorations
- Adhesive for indirect restorations

Contraindications

The use of the product is contraindicated if the patient is known to be allergic to any of its ingredients.

Limitations of use

- The product must not be used if the stipulated working technique cannot be applied.
- Dual-curing materials always have to be light-cured.
- When using the product, the 3s PowerCure curing program must not be employed.
-  The applicators are for single use only and must not be reused due to hygienic reasons.

Side effects

In rare cases, components of the product may lead to sensitization. In these cases, the material should no longer be used.

Interactions

- Peroxide-based bleaching agents and disinfectants may inhibit the polymerization of adhesives. Such materials must not be used ≤ 14 days prior to the application of the product.
- In order to achieve sufficient adhesion to enamel, the acid-etch technique must be employed. Contamination of the etched and dried enamel surface with e.g. saliva or blood will hamper this effect.
- Phenolic substances (e.g. eugenol) may inhibit the polymerization of the product.
- Before any treatment with the product, completely rinse off all iron- and aluminium-based styptic agents with water.

Clinical benefit

- Reconstruction of chewing function
- Restoration of esthetics

Composition

Syntac Primer: Triethylene glycol dimethacrylate, polyethylene glycol dimethacrylate, maleic acid, acetone in an aqueous solution.

Syntac Adhesive: Polyethylene glycol dimethacrylate and glutaraldehyde in an aqueous solution.

Heliobond: Bis-GMA, triethylene glycol dimethacrylate.

2 Application

2.1 Isolation

Ensure adequate relative or absolute isolation.

2.2 Pulp protection / Cavity lining

In very deep cavities, areas close to the pulp must be selectively coated with a calcium hydroxide liner and subsequently covered with a pressure-resistant cement (e.g. a glass ionomer cement).

2.3 Conditioning with phosphoric acid gel

Please observe the Instructions for Use of the phosphoric acid gel.

- Apply phosphoric acid gel to the prepared enamel first, and then to the dentin. The etchant should be left to react on the enamel for 15 to 30 seconds, and on the dentin for 10 to 15 seconds.
- **Note:** Do not overdry the dentin.
Following this, remove all etchant gel with a vigorous water spray for at least 5 seconds. Excess moisture should be removed leaving the dentin surface with a slightly glossy wet appearance (wet bonding).
This can be done with either an air syringe, a dry brush, a foam pellet or other lint-free absorbents.

2.3 Dosage

Note: Do not mix Syntac Primer and Syntac Adhesive. Apply the products sequentially!

Note: The solvent contained in Syntac Primer (acetone) is highly volatile. If this solvent evaporates, the monomer separates at 23 °C/ 73 °F and a mean drop size after about 60 seconds. Therefore, the primer should be dispensed from the bottle immediately before application. If several cavities are being restored at the same time, fresh primer should be dispensed for each cavity. Do not use the primer after the mentioned separation process has taken place.

- Dosage depends on the area of application.
- Dispense the required amount of material into a mixing well.
- Pick up and apply the product using a customary disposable applicator.
- Seal the bottle tightly after each use.

2.4 Application of Primer, Syntac Adhesive and Heliobond

- Apply Syntac Primer with the applicator on the cavity and gently rub in. Syntac Primer requires a contact time of at least 15 seconds on the dentin.
- Disperse excess Syntac Primer and dry thoroughly. Do not rinse.
- Apply Syntac Adhesive, leave it for 10 seconds and thoroughly dry the cavity with an air syringe. Do not rinse.
- Apply Heliobond and blow to a thin layer.
- **Note:** At this point, Heliobond must be light-cured when direct fillings are placed. For the placement of indirect restorations, Heliobond is polymerized together with the luting material.

2.5 Curing

Please observe the Instructions for Use of the curing light employed.

Note: The light probe should be held less than 5 mm above the product surface and should not touch unpolymerized material.

Note: If the diameter of the light guide tip is smaller than the restoration diameter, perform an overlapping polymerization in order to ensure that all areas of the restoration are covered.

- If used in combination with light-curing luting composites, the product does not require separate curing.
- For direct restorations, separate curing of the product is required:
- Light-cure it for 10 seconds at a light intensity of between 500 to 1,300 mW/cm².

2.6 Application of the filling or luting material

The filling material or the luting material must be applied according to the instructions of the manufacturer.

Application notes

- Syntac does not contain any light initiators. Therefore, the subsequent application of the light-curing bonding agent Heliobond is mandatory.
- When seating lab-fabricated inlays, crowns or veneers with low-viscosity, light- or dual-curing luting composites, Heliobond should not be light-cured separately. Heliobond should be polymerized together with the luting composite to assure the accurate fit of the restoration.
If self-curing composites are used, the light-curing bonding agent Heliobond must be applied. The bonding agent must be light-cured before the self-curing composite is applied.
- Prevent any contamination of the applicator with blood, saliva, sulcus fluid or water during application. In the event of contamination, the cavity must be rinsed again and the procedure must be repeated with a new applicator.

3 Safety information

- The product is an irritant. Contact with the skin, mucous membrane or eyes should be avoided.
- Syntac Primer is an irritant and highly flammable.
- In case of skin contact, immediately wash with copious amounts of water.
- If the material should come in contact with the eyes, immediately rinse with copious amounts of water and consult a physician or ophthalmologist.
- In rare cases, contact with the skin may cause sensitization to the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.
- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclar.com and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website (www.ivoclar.com).
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.
- Basic UDI-DI: 76152082ABOND007HW

Warnings

- Observe the Safety Data Sheet (SDS) (available in the download section of the Ivoclar Vivadent AG website www.ivoclar.com).
- Warnings for phosphoric acid gel: Phosphoric acid gel is corrosive. Avoid contact with the eyes, mucous membrane and skin (goggles are recommended for both patients and operators). If the material should accidentally come in contact with the eyes, immediately rinse with copious amounts of water and consult a physician / ophthalmologist. Observe the Safety Data Sheet (SDS) (available at www.ivoclar.com).

Disposal information

Remaining stocks must be disposed of according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks.

The following clinical residual risks are known:

- Failure of the adhesive bond
- Postoperative sensitivity

4 Shelf life and storage

- Close bottles immediately after use; exposure to light causes premature polymerization.
- Storage temperature: 2-28 °C (36-82 °F)
- If the indicated storage temperature is exceeded, proper functioning of the product cannot be ensured.
- Do not use the product after the indicated date of expiration.
- Expiry date: See note on packaging.
- Before use, visually inspect the packaging and the product for damage. In case of any doubts, please contact Ivoclar Vivadent AG or your local dealer.

5 Additional information

Keep material out of the reach of children!

Not all products are available in all countries.

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the product for its suitability and use for any purpose not explicitly stated in the Instructions.

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