Instructions for use

myQuickmat

All-round introkit

Circumferential
matrix
systems

en - ENGLISH	2
de - DEUTSCH	4
fr - FRANÇAIS	6
it - ITALIANO	8
es - ESPAÑOL	10
pt - PORTUGUÊS	12
no - NORSK	14
et - EESTI KEEL	16
fi - SUOMI	18
sv - SVENSKA	20

da - DANSK	22
cs - ČEŠTINA	24
tr - TÜRKÇE	26
el - ΕΛΛΗΝΙΚΑ	28
bg - БЪЛГАРСКИ	30
hu - MAGYAR	32
lv - LATVIEŠU VALODA	34
lt - LIETUVIŲ KALBA	36
ro - LIMBA ROMÂNĂ	38
sk - SLOVENČINA	40

Medical Devices Kit for professional use only
All individual components are CE marked
Circumferential matrix systems

Instructions for use

Description

myQuickmat All-round is the circumferential matrix system designed to perform anatomical posterior restorations of teeth with missing structures. The easy-to-use Quickmat All-round matrices enable restorations with naturally contoured anatomies and tight contact surfaces. Thanks to API (Autoclavable Placement Instrument), positioning Quickmat All-round matrices is stable and straightforward. API provides support and prevents the tensioning spool from tilting during matrix tightening. With myQuickmat All-round circumferential matrix system, you can comfortably treat challenging cases in the posterior region, working on an obstacle-free, accessible operating field.

Composition

API (Autoclavable Placement Instrument for Quickmat All-round): Thermoplastic
Quickmat All-round circumferential preloaded: thermoplastic and stainless steel | The Wedge: thermoplastic

Indications

myQuickmat All-round circumferential matrix system is indicated for Class II molar and premolar restorations, missing adjacent teeth, teeth with missing cusps, tooth isolation for medium and large build-ups and mesial-occlusal-distal cavity restorations.

Contraindications

None known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to myQuickmat All-round Introkit is necessary.

Preservation and storage


Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. In case of accidental inspiration of a myQuickmat All-round Introkit component (Circumferential preloaded matrices or wedge) seek immediate medical attention. In case of ingestion of a component of myQuickmat All-round Introkit, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All reusable products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before each use. Please refer to the Cleaning, Disinfection and Sterilization Instructions at polydentia.ch.

Application

① Select the appropriate matrix size according to the clinical situation. ② Place the spool on API (Autoclavable Placement Instrument for Quickmat All-round). ③ Position the spool over the tooth to be restored. ④ Gently push the matrix to insert it into proximal toward cervical. ⑤ Turn the tensioning knob clockwise to tighten the matrix around the tooth to be restored. Then, gently remove API. ⑥ Select the appropriate sized The Wedge to adjust the matrix cervically and insert it firmly. ⑦ Burnish the matrix. For a better contact, burnish the band towards the adjacent tooth using an appropriate instrument. Proceed with the restoration. ⑧ After the cavity is filled and light cured, turn the tensioning knob counterclockwise to loosen the matrix, then remove the matrix gently by pulling the spool upwards.

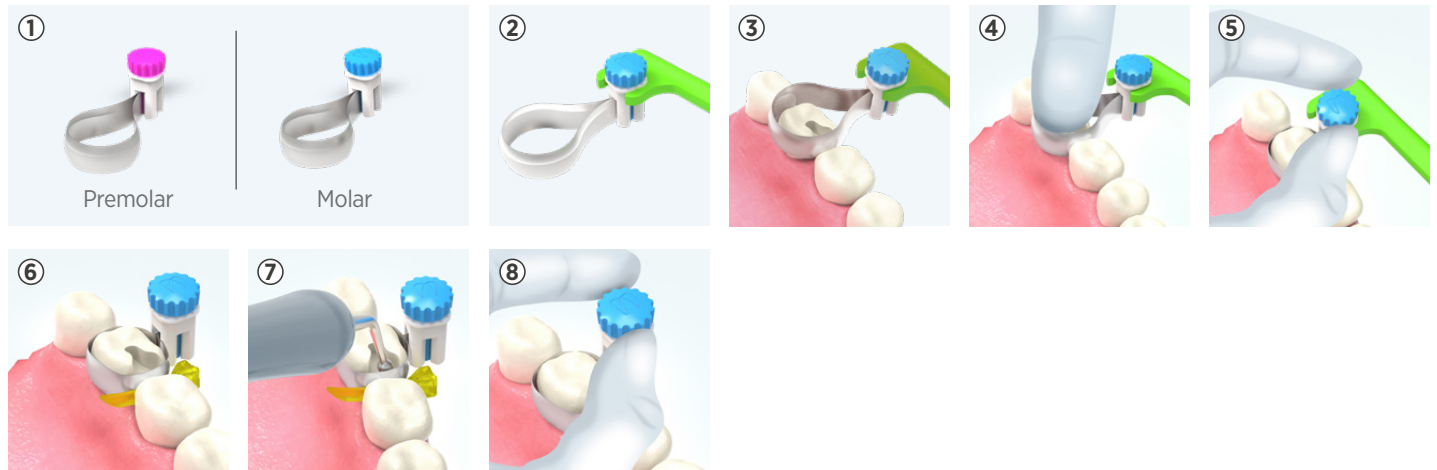Disposal

Because of their composition, matrices and wedges can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.

API (Autoclavable Placement Instrument) reconditioning

For a proper, effective and safe reconditioning of API (Autoclavable Placement Instrument) read the Cleaning, Disinfection and Sterilization instructions at polydentia.ch.

Clinical Applications

- Missing adjacent tooth
- Teeth with missing cusps
- Molar and premolar class II restorations
- Tooth isolation for core build-up
- MOD's

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