VOCO

Grandio®SO Light Flow

Instructions for use

MD EU Medical device

In accordance with DIN EN ISO 4049

Product description:

GrandioSO Light Flow is a highly flowable, light-curing, radiopaque nano-hybrid restorative material. It features a low viscosity and a high filler content at the same time. The material comes with two different application tips, a normal one (type 44) and a superfine one (type 45). The superfine application tip is especially suitable for accurate and precise applications.

GrandioSO Light Flow contains 76 % w/w inorganic fillers and is used with a dentine enamel bond.

Shades:

A1, A2, A3, A3.5, B1, OA2, WO, Incisal

Indications:

- Filling / characterisation of small cavities
- Filling / characterisation of class III, IV and V cavities
- Extended fissure sealing
- Blocking out undercuts
- Lining or coating cavities
- Repairing fillings, veneers and temporary restorations
- Luting translucent prosthetic pieces

Contraindications:

GrandioSO Light Flow contains methacrylate and BHT. **GrandioSO Light Flow** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

GrandioSO Light Flow is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

GrandioSO Light Flow should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Clean the teeth being treated. If applicable, mark occlusal contact points. Bring material to room temperature before use.

Shade selection:

To select a shade, compare the enclosed shade guide with the cleaned tooth before anaesthesia. Moistened the shade sample and tooth and select the shade, preferably in daylight.

Cavity preparation:

In principle, the cavities should be prepared according to the rules for adhesive restorative treatment; preparation should be minimally invasive in order to protect healthy dental hard tissue. For anterior restorations, angle off the enamel margins. Then clean and dry the cavity. Caries-free lesions in the cervical area do not need to be prepared; thorough cleaning is sufficient in this case.

Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

Matrix placement:

Take appropriate measures for areas where the applied **GrandioSO Light Flow** can flow out. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and placement of the matrix.

Pulp protection:

A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

Bonding materials:

GrandioSO Light Flow is applied with a dentine enamel bond, using the adhesive technique. Any light-curing bonding materials can be used. Observe the relevant instructions for use during preparation (etching technique) and processing.

For the repairing procedure of restorations made from materials like composite, glass ceramic, aluminium oxide ceramic, zirconium oxide ceramic or metal and the need for a primer please follow the instructions for use of the applied adhesive.

Application:

Lock the enclosed application tip on the syringe with a clockwise turn and directly apply **GrandioSO Light Flow**. Using the superfine application tip the material flows homogeneously upon moderate pressure on the plunger. For fillings with a thickness of over 2 mm, apply and cure in layers. Do not pull the plunger back on the syringe during or after use to guarantee the function of the non-dripping syringe.

The attached application tip is intended for single use only. Remove it after use and seal the syringe again tightly with its original cap. The use of a protective barrier is recommended to protect the syringe against contamination with bodily fluids, dirt from hands or oral tissue.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. Curing time if an LED/halogen lamp with a light output of at least 500 mW/cm² is used is

20 s for the shades A1, A2, A3, A3.5, B1, Incisal

40 s for the shades OA2, WO

Bring the light aperture as close as possible to the light-curing unit as otherwise the curing will be less effective. Inadequate curing can cause discolouration and complaints.

Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g., fine or extra-fine diamond burs, polisher). The tooth should be fluoridated as a final step.

Warnings, precautionary measures:

- Phenolic substances, especially products containing eugenol and thymol, lead to curing failures in restorative composites. Consequently, the use of zinc oxide/eugenol cements or other materials containing eugenol should be avoided in combination with restorative composites.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Barium aluminium borosilicate glass, silicon dioxide, HEDMA, BisGMA, TCDDMA, fumed silica, TEGDMA, BisEMA, initiators, stabilisers, pigments

Storage instructions and application method:

Storage at 4°C–23°C. Seal the syringes immediately after removing the material in order to avoid exposure to light and subsequent polymerisation. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **GrandioSO Light Flow** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **GrandioSO Light Flow** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 AA2586 E1 0322 99 © by VOCO