VOCO

VisCalor® bulk

CE0482

EN

Instructions for use

MD EU Medical device

In accordance with DIN EN ISO 4049

Product description:

VisCalor bulk is a light-curing, radiopaque nano-hybrid restorative material. It is characterised by an increased depth of cure while at the same time showing reduced shrinkage stress, which means that it can be cured in layers of 4 mm thickness.

VisCalor bulk contains 83 % w/w inorganic fillers and is used with a dentine-enamel bond.

VisCalor bulk is pre-warmed before application. The increase in temperature leads to reduced viscosity and thus effortless insertion into the cavity as well as very good adaptation of the composite to the cavity walls. When filling of the cavity is complete, viscosity increases again as the composite cools down. In this state the material is very easy to sculpt.

Shades:

Universal (U), A1, A2, A3

- Indications:**
- Class I and II posterior restorations
 - Base in class I and II cavities
 - Class V restorations
 - Locking, splinting of loose teeth
 - Repairing veneers, enamel defects and temporary C&B-materials
 - Extended fissure sealing
 - Restoration of deciduous teeth
 - Core build-up

Contraindications:

VisCalor bulk contains methacrylate and BHT. **VisCalor bulk** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

VisCalor bulk is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

VisCalor bulk should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Clean the teeth to be treated. If required mark occlusal contact points. Bring a preheating device for composites (e.g. **Caps Warmer**) to 68° C in accordance with the operating instructions. Insert caps and allow them to warm up (approx. 3 min.). Leave in the preheating device for max. one day. Alternatively, a **VisCalor Dispenser** can be used. Please refer to its accompanying instructions for use.

Shade selection:

Clean the teeth prior to shade selection. The shade is selected in comparison with the tooth while it is still moist.

Cavity preparation:

In principle, cavity preparation should be carried out according to the rules of adhesive filling therapy and should be minimally invasive to conserve healthy tooth substance. Clean and dry the cavity before treatment. Non-carious cervical lesions do not have to be prepared; thorough cleaning is sufficient here.

Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

Matrix placement:

Use of matrices is recommended for cavities with an approximal portion. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and insertion of the matrix.

Pulp protection:

A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

Bonding material:

VisCalor bulk is applied with a dentine enamel bond, using the adhesive technique. Any light-curing bonding materials can be used. Observe the relevant instructions for use during preparation (etching technique) and processing.

Application of VisCalor bulk:

After warming it in the **Caps Warmer**, insert the Cap into the Applicator and apply the material within 20 s. After 20 s, **VisCalor bulk** still has a temperature of 61 °C. When using the **VisCalor Dispenser**, select programme 1 (65 °C), which has a warm-up time of 30 s. The material is then kept warm for 2 min 30 s. Please also refer to the instructions for use of the **VisCalor Dispenser**. Insert **VisCalor bulk** directly into the prepared cavity, starting at the lowest point, and fill the cavity from bottom to top while keeping the tip of the Caps submerged. Ensure that the material is applied in a slow and steady flow to prevent air bubbles from forming. Apply **VisCalor bulk** in layers that are a maximum of 4 mm thick, adapt with a suitable instrument and light-cure afterwards.

Light-curing:

Conventional polymerisation devices with a wavelength in the range of 400 - 500 nm are suitable for light-curing the material. Depending on the light output and shade the curing time for increments of **2 - 4 mm** is:

LED-/halogen light	Universal (U)	A1, A2, A3
≥ 1000 mW/cm²	10 s	20 s
≥ 500 mW/cm²	20 s	40 s

The curing time for increments of **up to 2 mm** is 10 s for all shades when using an LED-/halogen light with a minimum light output of **500 mW/cm²**. Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discolouration and discomfort.

Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g., fine or extra-fine diamond bur, polisher). The tooth should be fluoridated as a final step.

Warnings, precautionary measures:

- Phenolic substances, especially products containing eugenol and thymol, lead to curing failures in restorative composites. Consequently, the use of zinc oxide/ eugenol cements or other materials containing eugenol should be avoided in combination with restorative composites.
- Each **VisCalor bulk** Caps unit should be used for one patient only. Caps should only be heated once (**VisCalor Dispenser**). Caps should be applied exerting uniform but not excessive pressure.

- Extract **VisCalor bulk** after warming in a preheating device, as high extrusion forces occur in the unwarmed state. The device must be allowed to cool down completely (approx. 2-3 minutes) between two uses with the **VisCalor Dispenser** so as to avoid overheating.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Barium aluminium borosilicate glass, silicon dioxide, BisGMA, TCDDMA, initiators, stabilisers, colour pigments

Storage:

Storage at 4°C-28°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **VisCalor bulk** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **VisCalor bulk** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 AA6065 E1 0125 99 © by VOCO