VOCO

Arabesk

Table 1

Light intensity (LED-/halogen lamp)	Polymerisation time (per layer)
≥ 500 mW/cm²	40 - 60 s
≥ 800 mW/cm²	20 s

Instructions for use

MD EU Medical device

In accordance with DIN EN ISO 4049

Product description:

Arabesk is a universal, light-curing, radiopaque micro hybrid dental composite filling material for the anterior and posterior areas, as well as for the inlay technique. It contains 60% by volume (76.5% by weight) inorganic fillers, microfillers (approx. 0.05 µm) and small particle fillers (approx. 0.5 - 2 µm). **Arabesk** can be polished to a high gloss and is distinguished by its high level of stability and colour stability. **Arabesk** is polymerised by halogen light (blue light) or LED-light. **Arabesk** is available in practical syringes for direct application.

Shades:

A1, A2, A3, A3.5, B2, B3, I

Indications:

- Fillings of class I, II, V in the posterior area which are not exposed to occlusal forces
- Reconstruction of traumatically damaged anteriors
- Facing of discoloured anteriors
- Shape and shade corrections to improve aesthetic appearance
- Locking of loosened anteriors
- Extended fissure sealing for molars and premolars
- Fillings of classes III, IV, V in the anterior area
- Repair of veneers
- Core build-ups for crowns
- Especially suitable for composite inlays

Contraindications:

Arabesk contains methacrylate and BHT. **Arabesk** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Arabesk is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Arabesk should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Clean the teeth being treated. If applicable, mark occlusal contact points. Bring material to room temperature before use.

Shade selection:

Clean the teeth prior to shade selection. The shade is selected in comparison with the tooth while it is still moist.

Cavity preparation:

In principle, the cavities should be prepared according to the rules for adhesive restorative treatment; preparation should be minimally invasive in order to protect healthy dental hard tissue. For anterior restorations, angle off the enamel margins. Then clean and dry the cavity. Caries-free lesions in the cervical area do not need to be prepared; thorough cleaning is sufficient in this case.

Ensure that the working field is sufficiently dry. The use of a rubber dam is recommended. It is recommended that shaping aids be used in cases where a portion of the cavity is in the proximal region. It helps to use translucent matrices, wedged in the proximal region. Keeping separation to the minimum makes it easier to shape the proximal contacts and attach a matrix. Suitable protection for the pulp should be applied close to the pulp and covered with a layer of stable cement if necessary.

Bonding materials:

Arabesk is applied with a dentine enamel bond, using the adhesive technique. Any light-curing bonding materials can be used. Observe the relevant instructions for use during preparation (etching technique) and processing.

Application:

Apply the selected shade of **Arabesk** (see shade selection) in layers of no more than 2 mm thickness; adapt using a suitable instrument and then cure.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. The polymerisation time per layer (Polymerisation time) as a function of the light intensity (Light intensity) can be found in Table 1 (Table 1). Bring the light aperture as close as possible to the light-curing unit as otherwise the curing will be less effective. Inadequate curing can cause discolouration and complaints.

Finishing:

After removing the matrices, the filling can be trimmed and polished immediately while being cooled (e.g. fine or extra-fine diamond burs, polisher). The tooth should be fluoridated as a final step.

Arabesk can be used for direct and indirect inlays according to the known methods. The physical stability values can be improved by the usual external (additional) curing.

Warnings, precautionary measures:

- Phenolic substances, especially preparations containing eugenol or thymol, inhibit the curing of **Arabesk**. Therefore, do not use zinc oxide eugenol cements or other eugenol-containing substances in combination with **Arabesk**.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Barium aluminium borosilicate glass, BisGMA, fumed silica, UDMA, TEGDMA, initiators, stabilisers, pigments

Storage instructions and application method:

Storage at 4°C-28°C. Seal the syringes immediately after removing the material in order to avoid exposure to light and subsequent polymerisation. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Arabesk** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Arabesk** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 AA1338 E1 0124 99 © by VOCO