Cention® Forte

[en] Instructions for Use

Polymer-based dental restorative material
(intra-oral dual-curing)

Cention® Primer

[en] Instructions for Use

Self-curing primer for Cention Forte

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Patient target group

- Patients with permanent teeth
- Patients with deciduous teeth

Intended users / Special training

- Dentists
- No special training required.

Use

For dental use only.

Description

Cention® Forte is a self-curing, radiopaque ($\geq 200\%$ Al) filling material with light-curing option, which releases fluoride, calcium and hydroxide ions (according to ISO 4049:2019 Type 1, Class 3).

It is suitable for use as a full volume replacement material in the restoration of Class I, II and V cavities. Cention Forte is also suitable for restoring occlusal surfaces.

100% aluminum has a radiopacity equivalent to that of dentin and 200% aluminum is equivalent to enamel.

Cention Forte can be optionally cured with light in the wavelength range of 400 – 500 nm.

Cention® Primer is a self-etching, self-curing primer for tooth structure. It is especially designed for use with Cention Forte.

Indications

Missing tooth structure in anterior and posterior teeth

Types of restorations:

- Suitable for Class I (occlusal) and II (occlusal and proximal) restorations in conjunction with Cention Primer;
- Class V restorations (cervical) in permanent teeth in conjunction with Cention Primer, when light-cured at a light intensity $< 1,300 \text{ mW/cm}^2$.

Contraindications

The use of the product is contraindicated if the patient is known to be allergic to any of its ingredients.

Limitations of use

- When used without Cention Primer.
- Due to esthetic reasons, Cention Forte is not suitable for Class III and IV restorations.
- If a dry working field cannot be established.
- If the stipulated working technique cannot be applied.
- When used as a luting composite.
-  If Cention Forte / Cention Primer is applied directly in the mouth of the patient, the capsule as well as the applicator should only be used for one patient due to hygienic reasons (prevention of cross-contamination between patients).
- Light intensities $> 1,300 \text{ mW/cm}^2$ must not be used for deciduous teeth.

Side effects

In rare cases, ingredients of Cention Forte / Cention Primer may cause a sensitizing reaction. The products must not be used in such cases.

In order to avoid irritation of the pulp, provide areas close to the pulp with suitable pulp/dentin protection. Selectively apply a calcium hydroxide-based preparation in areas close to the pulp and cover with a suitable cavity liner, if necessary.

Interactions

Peroxide-based bleaching and disinfecting agents may inhibit the polymerization of certain components of Cention Forte and Cention Primer. Materials of this kind must not be used ≤ 14 days prior to the treatment with Cention Forte & Cention Primer. Phenolic substances (e.g. eugenol) may also inhibit the polymerization of the material. Iron- or aluminium-based haemostatic agents must be rinsed away completely with plenty of water prior to the treatment with Cention Forte. Fluoride preparations must not be used prior to the treatment with Cention Forte. Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

Clinical benefit

Reconstruction of chewing function

Composition

Cention Forte:

UDMA, Calcium fluoro-silicate glass, aromatic-aliphatic UDMA, copolymer, barium-aluminium silicate glass, calcium-barium-aluminium fluoro-silicate glass, DCP, PEG-DMA, barium glass, and ytterbium trifluoride.

Total content of inorganic fillers: 58-59 vol%.

Particle size of the inorganic fillers: between 0.15 and 1.7 μm .

Cention Primer:

HEMA, Bis-GMA, MDP, ethanol, water, D3MA, methacrylate-modified polyacrylic acid, silicon dioxide and campherquinone

2 Application

I. Isolation

Adequate relative or absolute isolation is required.

II. Cavity preparation

The cavity is prepared according to tooth-preserving principles. Do not prepare sharp, internal edges and angles. Do not prepare additional undercuts in caries-free areas. The geometry of the cavity is pre-determined by the dimensions of the caries lesion or the old filling. Slightly bevel or round out the enamel margins using finishing diamonds (25 – 40 μm). Rinse with water spray to remove all residue in the cavity. Dry the cavity with water- and oil-free air.

Note: Do not overdry the dentin.

III. Pulp protection / Base

In very deep cavities only, areas close to the pulp should be selectively coated with a calcium hydroxide liner.

IV. Placement of matrix / interdental wedge

Use a wrap-around matrix for cavities affecting the proximal area or a sectional matrix band and secure it with wedges.

V. Activation of Cention Primer

- Dispense a drop of Cention Primer into a dish and protect it from light. Carefully close the bottle after use.
- Dip the corresponding coated Cention Primer single-use applicator in the liquid and stir through for approx. 5 seconds. The mixing process is completed when the flocked applicator has a uniform yellow colour. If this is not the case, the mixing process should be repeated.
- The primer must be used within 120 seconds of mixing.
- A new Cention Primer single-use applicator must be used for each new application.

Note:

The activated Cention Primer single-use applicator is indicated for only one patient due to hygienic reasons (prevention of cross-contamination between patients). Any remaining activated Cention Primer must be discarded after the treatment!

VI. Application of Cention Primer

Starting in the enamel, completely coat the tooth surfaces requiring treatment and scrub the primer in for a least 10 seconds. This interval must not be shortened. Simply spreading Cention Primer on the tooth surface is inadequate. Disperse Cention Primer with water- and oil-free compressed air until a shiny, thin, immobile film has formed.

Note:

Any contamination of the Cention Primer single-use applicator with blood, saliva, sulcus fluid or water during the application process must be avoided. If the cavity is contaminated, it must be rinsed again and the process must be repeated.

VII. Use of Cention Forte

- In order to activate the capsule, press the plunger on a flat surface. Make sure that the plunger is pressed in completely to ensure thorough mixing of the powder and liquid. The capsule must not be activated in the applicator! Immediately after activation, insert the capsule into the capsule mixer (4000 – 4800 rpm) and mix for 15 ± 2 seconds. The mixing time (Mixing time) is dependent on the temperature (Room/storage temperature). For the relevant recommendations, see table 1 (Table 1).
- After mixing, immediately remove the capsule from the device and place it into the applicator. Repeatedly activate the dispensing lever of the applicator until you can see the material in the transparent applicator tip. Then position the tip on the cavity floor and extrude the material evenly into the cavity. Keep the tip immersed in the material to prevent the formation of air bubbles. Subsequently, carefully pack and condense the material, remove occlusal excess and shape and contour the filling. The working time from the start of mixing is 2 minutes.

Working and setting times

Mixing time	– 15 ± 2 seconds
Dispensing time after mixing	– 20 seconds
Working time from the start of mixing	– 2 minutes
Setting time from the start of mixing	– 6,5 minutes

- The indicated working and setting times were determined under simulated clinical conditions. High temperatures during mixing decrease the working / setting time and dispensing time; low temperatures increase it.
- When using the self-curing mode, the matrix band must only be removed after the setting time has elapsed; then finishing can be started.
- The filling can be optionally light cured to a depth of 4 mm to speed up the curing process. For the recommendations regarding exposure time (Exposure time) per increment and light intensity (Light intensity) see table 2 (Table 2).
- Press the release button and pull the capsule upwards to remove it from the applicator.

VIII. Finishing / Checking the occlusion

Remove any excess material with tungsten carbide or diamond finishers after polymerization. Check the occlusion and articulation and apply appropriate grinding corrections to prevent premature contacts or undesired articulation paths on the surface of the restoration. If needed, use polishers as well as polishing discs and polishing strips to polish the restoration to a high gloss.

Application notes

- Cention Forte always has to be used with Cention Primer.
- In the case of adjustments, additional Cention Forte can be directly applied to the polymerized material. If the inhibition layer has already been removed, the filling surface must be roughened and then coated with Cention Primer before fresh Cention Forte can be applied.
- Use Cention Forte at room temperature. Cold components may be difficult to mix.

3 Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/ Liechtenstein, website: www.ivoclar.com, and your responsible competent authority.
- The current Instructions for Use are available on the website (www.ivoclar.com).
- Explanation of the symbols: www.ivoclar.com/eIFU
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.
- Basic UDI-DI: 76152082AFILL016JP

Warnings

- Observe the Safety Data Sheet (SDS) (available at www.ivoclar.com).
- Prevent any contact of uncured Cention Forte and Cention Primer with skin / mucous membrane and eyes. If Cention Primer should accidentally come in contact with the eyes, immediately rinse with plenty of water and consult a physician or eye specialist. If the product comes in contact with the skin, wash with soap and plenty of water.
- Unpolymerized Cention Forte and Cention Primer may have a slight irritating effect and may lead to a sensitization against methacrylates.
- Customary medical gloves do not provide protection against the sensitizing effect of methacrylates.

Disposal information

Remaining stocks or removed restorations must be disposed of according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks.

The following clinical residual risks are known:

- Failure of the adhesive bond (loss of the filling)
- Inclusion of air bubbles during the placement of the filling
- Chipping, fractures
- Ingestion of material
- Postoperative sensitivity

4 Shelf life and storage

- Storage temperature: $2-28^{\circ}\text{C}$ / $36-82^{\circ}\text{F}$
- Do not use the product after the indicated expiry date.
- If the indicated storage temperature is exceeded, proper functioning of the product cannot be ensured. Keep the foil bag of the Cention Forte capsule closed until use.
- Cention Primer single-use applicators must be stored in the plastic container provided. Keep the plastic container closed.
- Expiry date: see information on syringes and packages.
- Before use, visually inspect the packaging and the product for damage. In case of any doubts, please contact Ivoclar Vivadent AG or your local dealer.

5 Additional information

Keep material out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

Table 1

Room/storage temperature	Mixing time
20-22 °C (68-72 °F)	17 s
22-26 °C (72-79 °F)	15 s
26-28 °C (79-82 °F)	13 s

Table 2

Light intensity	Exposure time
500-900 mW/cm ²	20 s
1,000-1,300 mW/cm ²	15 s
1,800-2,200 mW/cm ²	2 x 5 s *

* Suitable for Class I (occlusal) and II (occlusal and proximal) restorations in conjunction with Cention Primer

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