VivaDent® Aerosol Reduction Gel

To reduce spray mist and aerosols, for ultrasonic scaling

- Reduction* of aerosols produced by ultrasonic scalers by up to 99%
- Better visibility of the treatment area during ultrasonic scaling
- Increased patient comfort due to less aerosols on the face

Ingredients

Polyethylene oxide, Glycerol

Warnings

- Keep VivaDent Aerosol Reduction Gel out of the reach of children
- For dental use only!
- Do not fill the VivaDent Aerosol Reduction Gel directly into the coolant tank of the ultrasonic scaler

Note:

- When using ultrasonic scalers, please observe the local and national hygiene guidelines and the manufacturer's instructions
- The device must be rinsed with drinking water after ultrasonic scaling with the VivaDent Aerosol Reduction Gel
- Only suitable for ultrasonic scalers with an external coolant tank. Not suitable for air-polishing devices


* The reduction of aerosol is dependent on the amplitude of the scaler tip

VivaDent® Mixing Bottle

Easy handling: reusable bottle for mixing solutions

Container for mixing and storing solutions

Clean the VivaDent Mixing Bottle with conventional cleaning agents after each use

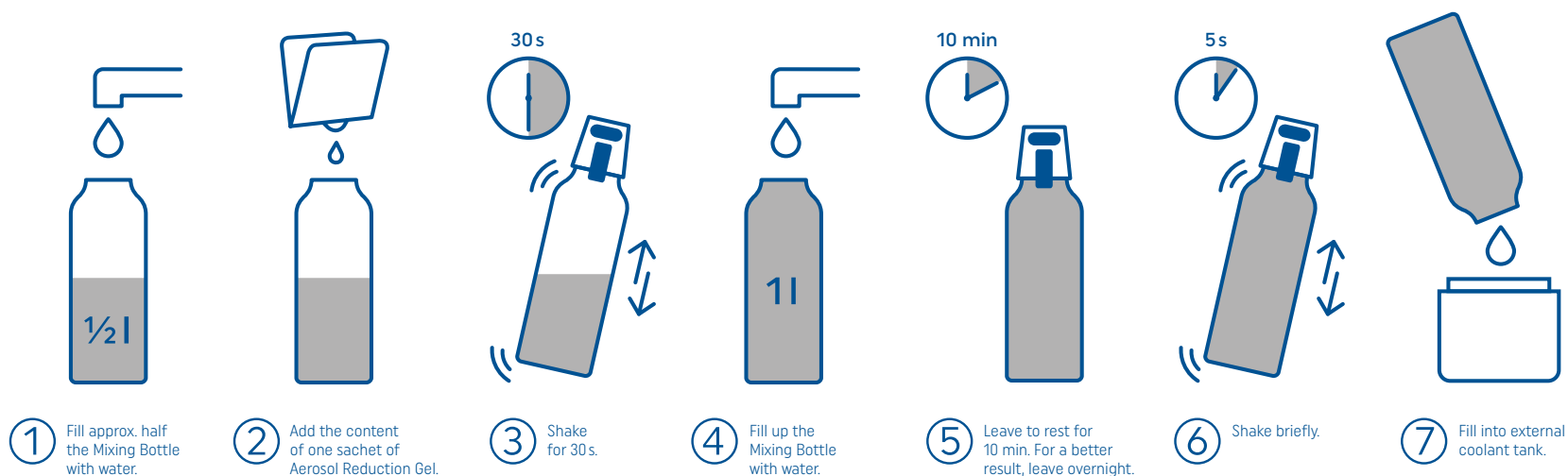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