VOCO Aqua Meron

EN Instructions for use
MD EU Medical device
In accordance with DIN EN ISO 9917-1

Product description:
Aqua Meron is a water-mixable glass ionomer cement for luting. Its outstanding features include low solubility in the mouth and a low level of acidity.

Indications:
For attaching crowns, bridges, inlays, onlays, posts and orthodontic bands, as well as for the conventional luting of high-strength full ceramic restorations as for example ceramics of silicates, zirconium oxide, aluminium oxide.

Contraindications:
Aqua Meron contains polyacrylic acid. **Aqua Meron** should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

Patient target group:
Aqua Meron is suitable for use in all patients without any age or gender restrictions.

Performance features:
The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:
Aqua Meron should only be used by a professionally trained dental practitioner.

Use:
Thoroughly clean and dry enamel, dentine and cavity surfaces. Clean cores before luting (e.g. pumice, water), thoroughly rinse and dry with a gentle stream of air or cotton pellets (do not overdry!). Ensure sufficient residual dentine thickness to prevent pulp irritation. If necessary, protect the pulp or build up the core.
Attention! **Aqua Meron** must be mixed only with water! Shake powder well before use!
Aqua Meron has to be applied at room temperature (15 °C - 25 °C). In case of higher temperatures it is recommended to use a cooled glass plate.
Place powder and water on a glass plate or mixing pad and work the powder portion by portion into the water. Use 2 drops of water for 1 level measuring scoop of powder (mixing ratio powder : water = 3.3 - 3.8 g : 1).
When dispensing the water hold the dropper bottle vertically, making sure that the metal canula protrudes approx. 2 - 3 mm above the top.

Mixing time	Working time	Setting time
30 s	3 min	3 - 4 min

The mixing time is approx. 30 s. **Aqua Meron** can be worked at room temperature for approx. 3 min after commencement of mixing and sets in the mouth within 3 - 4 min. Higher temperatures will shorten the working time and lower temperatures will prolong it.
In case of narrow bridges or crowns it is important to ensure that the surplus of **Aqua Meron** is not too great, nor its viscosity too high, since glass ionomer cements have specific flow properties.
Aqua Meron has reached the right consistency when it can be applied to the tooth cores and inside the crown with a stiff brush. During application and setting time prevent contact with water and saliva.

Warnings, precautionary measures:
– Avoid contact with mucosa and eyes!
– Do not use insulating materials either in order to maintain full adhesion to the dentine.
– Rinse metal instruments with cold water immediately after use and before any residual cement has time to set. After removal of orthodontic bands, cement residues can be removed gently and rapidly by means of an ultrasonic scaler.
– Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):
Calcium aluminium fluorosilicate glass, water, polyacrylic acid, tartaric acid

Storage instructions and application method:
Storage at 4°C - 23°C. Reclose well after use. Do not use after the expiry date.

Disposal:
Dispose of the product in accordance with local regulations.

Reporting obligation:
Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Aqua Meron** must be reported to VOCO GmbH and the responsible authority.

Note:
The Summary of Safety and Clinical Performance of **Aqua Meron** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 AA1172 E1 0323 99 © by VOCO