CE 0476

Mode d'emploi

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# CE 0476

## Instructions For Use

ENDO N2 is a root canal cement to be used exclusively by dentists.

### Composition:

1 g of powder contains:	
Zinc Oxide	656,80 mg
Basic Nitrate Bismuth	151,7 mg
Basic carbonate Bismuth	101 mg
Paraformaldehyde	40,0 mg
Titanium Dioxide	47,6 mg
Red Ferric-oxide	1 mg
Zinc Stearate	0,75 mg
Dehydrate Zinc Acetate	0,75 mg
Yellow Ferric-oxide	0,4 mg
1 g of liquid contains:	
Eugenol	770 mg
Peanut oil	200 mg
Rose oil	18 mg
Lavender oil	12 mg

### Contraindications:

The use of ENDO N2 is contraindicated in patients showing an assessed hypersensitivity to the product.

### Warning:

ENDO N2 is harmful if inhaled or swallowed or upon contact with the skin. Irritates eyes, skin, respiratory tract and the mucosa. The product must be used only by personnel specialized in dentistry or equivalent profession. Please use safety gloves, mask and glasses. A rubber dam must be used to avoid any accidental events such as swallowing the cement. The product does not present any generic contraindications, except for individual intolerances in patients with a proven sensitivity to any of the components. The product is not to be used in pregnant women. In the event of any changes to the organoleptic characteristics of the product (i.e. packaging of the powder), isolate the non-conforming packaging and dispose of as special waste. Prior to using ENDO N2 make sure the package is intact. An inadequate mechanical and chemical cleaning process of the root canal will subsequently affect endodontic procedures rendering the use of ENDO N2 ineffective.

### Dosage and directions for use:

Pour 3-4 drops of liquid ENDO N2 on a glass slide. Add small quantities of, ENDO N2 Powder until the 3-4 drops of liquid are absorbed. Using a spatula, mix until a creamy and homogeneous paste of average consistency is obtained.

### Canal Filling Technique:

After carefully reaming the root canal, fill the root canal using a spiral, having previously chosen the adequate size, as follows: the spiral, loaded with a small quantity of ENDO N2 paste, is inserted into the canal, without rotating it, until it reaches the deepest level of the canal preparation. To remove the spiral, slowly rotate in clockwise direction. If and when necessary, repeat the operation.

### Notes:

- As the canal has a minimum volume, it is, therefore, recommended to load the spiral each time with small quantities of ENDO N2. An exaggerated amount of the paste could block access to the canal and as such prevent the air inside the canal from escaping.
- If the apical foramen is open, should the operator make any uncontrolled movements, an unwanted ENDO N2 penetration could occur beyond the apex. Therefore, pay utmost attention in preventing this type of problem
- ENDO N2 is a root canal cement which, as for all root canal cements, must never be pushed beyond the apex.
- ENDO N2 is radiopaque and adheres excellently to the canal wall..

### Notes for correct preservation:

Should the wrapping be damaged or show signs of spilling, use safety gloves to remove it. This type of waste is to be considered special.

Do not use ENDO N2 beyond the expiration date printed on the package.

The package can be stored at room temperature (18°-25° C) as long as it is properly closed and protected from light and humidity. Even though it is not necessary, keeping the product refrigerated is recommended.

### Appearance:

The package contains:  
1 bottle contains 10 g of powder, 1 bottle contains 6 g liquid and 1 medicine dropper.  
The instructions for use were drawn up on 20/05/2015

### Notes about the symbols

- Lot number
- Expiration date (YEAR-MONTH)
- Read instructions for use
- Manufacturer
- Keep away from sunlight
- Keep dry
- Caution
- Store 18°-25° C

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This product is made to be used in the dental field and must be used according to the instructions for use. The manufacturer declines any responsibility for damage caused by different and inadequate use of the product. The user is responsible for personally checking the fitness of the product for the intended uses he has foreseen, especially if such uses are not indicated in the instructions for use.

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Hager & Werken GmbH & Co.KG