Prior to use, carefully read the instructions for use.

EN

Miracle Mix™

RADIOOPACQUE SILVER ALLOY-GLASS IONOMER CEMENT RESTORATIVE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Core build-up.
- Restoration of primary teeth.
- Cases where radiopacity is required.
- As a base or liner.
- Class I, limited Class II, temporary fillings, etc.

CONTRAINDICATIONS

- Pulp capping.
- Avoid use of this product in patients with known allergies to glass ionomer cement, silver, tin or copper.

DIRECTIONS FOR USE

- POWDER AND LIQUID DISPENSING
 Pour the Miracle Mix alloy into the bottle of Miracle Mix powder and shake thoroughly.
 a) The recommended powder to liquid ratio is 2 or 3 level scoops of powder to 2 drops of liquid.
 b) Shake the powder bottle occasionally.
 c) Hold the liquid bottle vertically and squeeze gently.
 d) Close bottles immediately after use.
 e) Shake the powder bottle occasionally.
- MIXING
 Dispense powder and liquid onto pad or glass slab. Divide the powder into 2 equal parts. Mix the first portion with all of liquid for 15-20 seconds. Incorporate the remaining portion and mix the whole thoroughly for 20 seconds.
 Note:
 a) Heavy consistency is recommended.
- RESTORATIVE TECHNIQUE
 a) Prepare tooth using standard techniques. Extensive mechanical retention is not necessary. For pulp capping, use calcium hydroxide.
 b) Wash and dry but DO NOT DESICCATÉ. CAVITY CONDITIONER or DENTIN CONDITIONER is recommended to remove the smear layer.
 c) Mix the required amount of cement. Working time is 1 min. 30 seconds at 23°C (73.4°F). Higher temperatures will shorten working time.
 d) Remove surface moisture but DO NOT DESICCATÉ.
 e) Transfer cement to the preparation using a syringe or suitable placement instrument. Avoid air bubbles.
 f) Form the contour.
 g) Setting time is 5'00" to 5'30" from start of mixing.
- FINISHING
 Use standard techniques.

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39.2-77.0°F).

PACKAGES

- Package: 15 g powder with measure, 10 g (8.0 mL) liquid, 17 g alloy
- Bottle of 15 g powder with measure
- Bottle of 10 g (8.0 mL) liquid
- Bottle of 17 g alloy

Note:
 All packaging types may not be available in all countries.

CAUTION

- In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- DO NOT mix powder or liquid with any other glass ionomer components.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gdc.dental. In this way you will contribute to improve the safety of this product.

Last revised: 06/2019

ИЗВЕЩАНИЕ О НЕЖЕЛАТЕМЫХ ЭФФЕКТАХ
Если вы становитесь свидетелем любого нежелательного эффекта, реакции или подобных событий, experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gdc.dental. In this way you will contribute to improve the safety of this product.

ИЗВЕЩАНИЕ О НЕЖЕЛАТЕМЫХ ЭФФЕКТАХ
Если вы становитесь свидетелем любого нежелательного эффекта, реакции или подобных событий, experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gdc.dental. In this way you will contribute to improve the safety of this product.

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8565, Japan
DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8565, Japan
EU: **GC EUROPE N.V.**
Researchpark Hasseltse-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL +32 16 74 100 00

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