## EN

### MACRO-LOCK POST® ILLUSION® X-RO® FOR DENTAL USE ONLY

#### DESCRIPTION

**MACRO-LOCK POST ILLUSION® X-RO®** are dental posts used for reconstruction of devitalized teeth. They are made of X-RO yellow translucent fibers bound in a colored resin Epoxy-vinyl ester matrix, which becomes translucent in the mouth. The post color will re-appear on command under cold air, water spray if re-treatment is required.

The **MACRO-LOCK POST ILLUSION® X-RO®** is a serrated-surfaced post, apically tapered with a length of 17,5 mm. The post has fluted slots around its axis for additional macro-retention to the adhesive used, by increasing the surface area available on the post for cement contact.

The radiopacity level of the **MACRO-LOCK POST ILLUSION® X-RO®** is 340% Al eqv. (ISO 4049)

The devices are available in 6 sizes and are used in association with an universal pre-shaping drill to perform a first hole and finishing drill associated to each post size according to the anatomical situation of the tooth using the radiograph and the plastic calibrating card.

1. Select the correct **MACRO-LOCK POST ILLUSION® X-RO®** size. Determine the post size according to the anatomical situation of the tooth using the radiograph and the plastic calibrating card.

2. Determine the length of the coronal build-up, which will account for 1/3 of the final post length. Remove the root filling with the **MACRO-LOCK STARTER DRILL** selected (rotation speed 800-1200 rpm) as deep as necessary for the post to be inserted to 2/3 of the root length. At least 4 mm of root canal filling should remain in the apical region. In case of curved canal, this calculated length is not suitable.

3. Shape the canal with the Finishing Drill corresponding to the selected **MACRO-LOCK POST ILLUSION® X-RO®** size, (rotation speed 1000-2000 rpm).

4. Try in the post, to check that the post is properly seated in the canal.

5. Shorten the post to its final length with a diamond disc outside of the mouth. Never use a crimping instrument, such as wire cutters, as the pressure can destroy the structure of the post.

6. Clean the post with alcohol.

7. Apply etchant in the post space and to the exposed dentin for 15 seconds\*. Rinse for 10 seconds. Remove the excess water with paper points but leave surface moist. Do not put the etchant in contact with the gingiva.

8. Apply 2 coats\* of the primer with a thin brush (e.g. Compositbrush RTD) in the post space. Remove excess using paper points and gently air dry all surfaces. Enamel and dentin surfaces should have a uniform, glossy appearance. If not, repeat application. Light-cure the primer\* inside the post space for 10 to 20 seconds.

9. Apply a single coat of primer\* to the post. Gently air dry 5 seconds and light-cure the post for 10 to 20 seconds (if needed) outside the mouth.

10. If using a dual cure resin cement for both cementation and core build up such as **Corecem™** (recommended technique), Mix the cement and apply it in the post space from bottom to the top using the root canal tip. Seal the post immediately. Remove excess cement with appropriate instruments. If using dual-cure cement, light-cure for 40 to 60 seconds, applying gentle pressure on the post with the tip of the curing light probe. Apply 2 coats of bonding agent\* to the exposed post(s), cement and invoked tooth structure. Remove excess and air-dry with jet of air. Light-cure for 20 seconds (if needed).

11.a. If using a self-etching cement\*, there is no need to etch and prime the post space. Mix the cement and apply it in the post space from bottom to the top using a root canal tip. Seal the post immediately. Remove excess cement. Light cure if needed.

11.c.i. After etching, rinsing, drying of involved tooth structure, apply 2 coats of bonding agent\* to the exposed post(s) and coronal part. Remove excess and air dry with jet of air. Light cure for 20 seconds (if needed).

12. Directly model the build-up using a composite core build-up material\*. RTD recommends that the coronal end of the post be covered by core composite.

#### COMPOSITION

**MACRO-LOCK POST ILLUSION® X-RO®** posts are fiber-reinforced polymer, made of X-RO fiber (90%) and epoxy resin matrix (20%) which contains colored pigment.

**MACRO-LOCK®** drills are made of stainless steel.

#### INTENDED USE

**MACRO-LOCK POST ILLUSION® X-RO®** posts have been designed to support and secure the coronal restoration in case of insufficient residual tooth structure (<4 mm).

**MACRO-LOCK®** drills are intended to prepare the tooth for insertion of a post that will support and secure the coronal restoration, where insufficient residual tooth structure (<4 mm) exists.

These products should only be used in hospitals, clinics or dental office with qualified dental personnel (dentists).

#### INDICATIONS FOR USE

**MACRO-LOCK POST ILLUSION® X-RO®** posts and associated drills are indicated in case of insufficient residual tooth structure (<4 mm) to support and secure the coronal restoration.

#### CONTRAINDICATIONS

**MACRO-LOCK POST ILLUSION® X-RO®** posts and associated drills are contraindicated if coronal tooth structure is less than 1.5 mm.

#### WARNINGS

None known to date.

#### PRECAUTIONS

There is no data for the use of **MACRO-LOCK POST ILLUSION® X-RO®** posts and associated drills for pregnant women and children under 18 years old (for posts only). It is not recommended for such use.

If the device is used in a way different than the one recommended in the DFU, loosening, breakage of the post or extraction of the tooth might occur.

#### MACRO-LOCK POST ILLUSION® X-RO® posts:

• The post must be immersed or soaked in Isopropanol for 1 minute and dried gently with air syringe for 15 s at least prior to bonding before insertion in the canal. Avoid touching the posts with your fingers after that.

The post can neither be sterilized nor disinfected in thermodesinfector and the use of autoclave is not recommended.

• Posts are single use devices. They must not be re-used to avoid the risk of contamination.

• Shortening the post with a diamond disc should be done outside of the mouth. Never use a crimping instrument, such as wire cutters, as the pressure can destroy the structure of the post.

• The use of rubber dam is recommended.

• The crown must be done the preparation of at least 1.5 mm of healthy dentin in order to get the ferrule effect.

#### MACRO-LOCK® drills:

• **MACRO-LOCK POST ILLUSION® X-RO®** drills are supplied non-sterile. Disinfected and sterilized instruments before each use. Disinfect instruments with disinfecting or cleaning agents adapted for rotary instruments\*. Do not disinfect instruments in thermodesinfector.

• Inspect instruments visually before each use: change to new drills when they are damaged, corroded or dull.

• The universal drill may be used only to remove root canal filling partially, and the finishing drills may be used only to prepare root canal.

• The use of rubber dam is recommended.

• The lifetime of use for the **MACRO-LOCK®** drills stated is 15 uses, following removal of dental surgeon is informed that the drill should be replaced.

• Drills must be sterilized individually in an autoclave with the following settings\*\*:

✓ Wrap individually in disposable sterilization pouches or tubing that comply with the regulation standard in force in the country of use,

✓ Autoclave: type B complying with regulation standard in force,

✓ Sterilization temperature: 134 °C – sterilization time: 18 minutes,

✓ Store the sterilized components in a dry, dust-free place,

✓ If the packaging's integrity appears to be compromised, before using again, place the component in a new pouch and re-sterilize according to the protocol described in the IFU.

#### ADVERSE REACTIONS

- Debonding of the post
- Root fracture
- Fracture of the post

#### STEP-BY-STEP INSTRUCTIONS

1. Root canal preparation: remove gutta-percha to preplanned depth with a Gates-Glidden, Peeso reamer or Largo. Radiographic verification is recommended.

2. Select the correct **MACRO-LOCK POST ILLUSION® X-RO®** size. Determine the post size according to the anatomical situation of the tooth using the radiograph and the plastic calibrating card.

3. Determine the length of the coronal build-up, which will account for 1/3 of the final post length. Remove the root filling with the **MACRO-LOCK STARTER DRILL** selected (rotation speed 800-1200 rpm) as deep as necessary for the post to be inserted to 2/3 of the root length. At least 4 mm of root canal filling should remain in the apical region. In case of curved canal, this calculated length is not suitable.

4. Shape the canal with the Finishing Drill corresponding to the selected **MACRO-LOCK POST ILLUSION® X-RO®** size, (rotation speed 1000-2000 rpm).

5. Try in the post, to check that the post is properly seated in the canal.

6. Shorten the post to its final length with a diamond disc outside of the mouth. Never use a crimping instrument, such as wire cutters, as the pressure can destroy the structure of the post.

7. Clean the post with alcohol.

8. Apply etchant in the post space and to the exposed dentin for 15 seconds\*. Rinse for 10 seconds. Remove the excess water with paper points but leave surface moist. Do not put the etchant in contact with the gingiva.

9. Apply 2 coats\* of the primer with a thin brush (e.g. Compositbrush RTD) in the post space. Remove excess using paper points and gently air dry all surfaces. Enamel and dentin surfaces should have a uniform, glossy appearance. If not, repeat application. Light-cure the primer\* inside the post space for 10 to 20 seconds.

10. Apply a single coat of primer\* to the post. Gently air dry 5 seconds and light-cure the post for 10 to 20 seconds (if needed) outside the mouth.

11.a. If using a dual cure resin cement for both cementation and core build up such as **Corecem™** (recommended technique), Mix the cement and apply it in the post space from bottom to the top using the root canal tip. Seal the post immediately. Remove excess cement with appropriate instruments. If using dual-cure cement, light-cure for 40 to 60 seconds, applying gentle pressure on the post with the tip of the curing light probe. Apply 2 coats of bonding agent\* to the exposed post(s), cement and invoked tooth structure. Remove excess and air-dry with jet of air. Light-cure for 20 seconds (if needed).

11.c.i. After etching, rinsing, drying of involved tooth structure, apply 2 coats of bonding agent\* to the exposed post(s) and coronal part. Remove excess and air dry with jet of air. Light cure for 20 seconds (if needed).

12. Directly model the build-up using a composite core build-up material\*. RTD recommends that the coronal end of the post be covered by core composite.

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