Palodent® V3

Sectional Matrix System

INSTRUCTIONS FOR USE ENGLISH 2

Système de Matrices Sectionelles

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Sistema de Matrices Seccionales

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Sistema de matrizes seccionais

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Sectional Matrix System

INSTRUCTIONS FOR USE – ENGLISH

Caution: This is a medical device. For dental use only.

USA: Rx only.

1. PRODUCT DESCRIPTION

The Palodent® V3 Sectional Matrix System employs a nickel-titanium retaining ring, interproximal wedge, WedgeGuard and sectional matrices for placement of restorations in the posterior region. Sectional matrices are suitable for use with all posterior restorative materials. Palodent® V3 EZ Coat matrices have a micro-thin, non-stick finish that minimizes the chance that bonding agent or composite sticks to the band, making matrix removal easier. The retaining rings are available in narrow, for most premolar and small molars or universal, for larger molars. Rings may be used either singly or in tandem, for simultaneous restoration of multiple surfaces.

The advantages of the sectional matrices and rings include natural contours for better control of contact areas and embrasures, ease of placement, better visualization of the operative field, and more comfort for the practitioner and patient. Eliminating the leverage of a conventional matrix retainer allows the practitioner to use less tension, avoiding excessive compression of tooth and easier formation of anatomical proximal contact areas. When placed prior to preparation, the innovative WedgeGuard combination wedge and shield protects approximating surfaces from inadvertent damage during preparation. Using the pin tweezers, the shield easily slides away, leaving the wedge securely in place.

1.1 Indications

- The Palodent® V3 System is indicated for Class II restorations to facilitate the contouring of a direct placement restorative materials.
- The Palodent® V3 WedgeGuard is indicated for pre-operative placement to protect surfaces adjacent to Class II, Class III, veneer and crown preparations.

1.2 Contraindications

None known.

1.3 Delivery forms (Some delivery forms may not be available in all countries)

- Sectional matrices are available in regular polished stainless steel or color-coded, non-stick Palodent® V3 EZ Coat in 5 sizes: 3.5 mm, 4.5 mm, 5.5 mm, 6.5 mm and 7.5 mm. These matrices are suitable for use with all posterior restorative materials.
- Retaining rings are available in narrow (dark blue) and universal (light blue) sizes suitable for use in the posterior region either singly or in tandem.
- Wedges and WedgeGuards are available in small (dark blue), medium (medium blue) and large (light blue) sizes.

1.4 Composition

- Sectional Matrices and WedgeGuard shield: Polished stainless steel
- Palodent® V3 EZ Coat matrices: Whitford 8820HR-coated polished stainless steel
- Retaining rings: Nickel-Titanium with Glass Fiber Reinforced Tines
- Wedge and WedgeGuard: Polyacetal Thermoplastic
- Placement forceps and pin tweezers: 420 Stainless steel

2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

- The Palodent® V3 System consists of small, sharp parts. As with all intraoral instruments, care must be exercised to ensure stable handling and placement. Placement of rubber dam or other suitable isolation is advised prior to operative procedures. Wear suitable protective eyewear and gloves.
 - Aspiration:** If aspiration of matrix, wedge, WedgeGuard, or retaining ring occurs seek immediate medical attention.
 - Ingestion:** If accidental swallowing occurs, encourage patient to seek medical assistance to locate swallowed part. If nausea or illness develop, seek medical attention immediately.
 - Laceration:** The Palodent® V3 matrices and WedgeGuard shields are sharp, and may cut unprotected skin and periodontal tissues. Avoid excessive apical pressure. Handling matrices and WedgeGuard shields with gloved hands and/or pin tweezers or similar instrument is recommended. In the event a clean matrix or WedgeGuard shield causes a skin laceration, employ usual first aid measures. Discard and properly dispose matrix or WedgeGuard shield.
- Used matrices and accessories are contaminated. Follow proper exposure control plans. In the event of an accidental exposure, follow recommended post-exposure procedures. Following use of the Palodent® V3 System, proper handling and disposal techniques are required. Used matrices are contaminated and sharp. When handling or disposing, use precautions as when handling or disposing of other contaminated sharps.
- Product is provided non-sterile. Components should be sterilized prior to clinical use (see Precautions, Step-by-Step Instructions, Hygiene and Disposal).
- Care should be taken to have good control on any instrument to protect patients from injury.
- Excessive opening force with forceps during ring placement or removal may compromise mechanical integrity, resulting in forceps or ring time fracture/separation. If fracture occurs, to prevent aspiration or ingestion retrieve all separated pieces and properly discard.
- When properly placed, retaining rings exert a clamping force for stabilization. Use of narrow rings on large prepared molars or use of any ring on teeth compromised by extensive preparation and/or with unsupported tooth structure may lead to tooth crazing or fracture. For compromised teeth, consider alternative matrix retainer.

2.2 Precautions

- This product is intended to be used only as specifically outlined in the Instructions for Use. Any use of this product inconsistent with the Instructions for Use is at the discretion and sole responsibility of the practitioner.
- Care must be taken when placing the retaining ring, due to the spring nature of the ring. Ensure ring is fully seated in forceps before placement. Before withdrawing forceps, check that the ring is seated securely apically (see Step-by-Step Instructions). Tie dental floss to the Palodent® V3 ring for added security.
- Excessive pressure on matrix, wedge, WedgeGuard, or retaining ring in an apical direction may result in injury to periodontal tissues (see Warnings).
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- Do not use matrices, wedges or accessory tools which have not been properly pre-use processed or reprocessed. Before use (and each re-use for reusable retaining rings, forceps and pin tweezers), all components must be properly cleaned and autoclaved to reduce the risk of cross contamination (see Hygiene and Disposal below).
- Do not use materials directly from cold storage. Allow at least 15 minutes for materials to reach room temperature before use to ensure optimum ring retention and separation.
- Do not use accessory tools other than those designed, supplied, and/or outlined for use with the Palodent® V3 System. Injury may result (see Warnings, Step-by-Step Instructions).
- Palodent® V3 forceps should not be used with other manufacturer's retaining rings. Injury may result (see Warnings, Step-by-step Instructions).

2.3 Adverse Reactions

- Product misuse may cause failure resulting in respiratory, digestive, eye, skin, or mucosa injury (see Contraindications, Warnings and Precautions).
- Failure to properly employ accepted handling and disposal techniques may present bio-hazard (see Warnings and Precautions).
- Use of narrow retaining rings on large teeth, or use of any retaining ring on weakened or unsupported tooth structure may lead to tooth fracture (see Warnings).

2.4 Storage conditions

Store at temperatures between 2 °C-36 °C/35 °F-96 °F, away from moisture or excessive humidity. Allow material to reach room temperature prior to use. Keep out of sunlight. Keep away from flame or source of ignition.

3. STEP-BY-STEP INSTRUCTIONS

1. Pre-operative Sterilization

Matrices, wedges and WedgeGuards are intended for single use and are not supplied sterile. These components should be steam autoclaved one time prior to clinical use. Retaining rings, forceps and pin tweezers are not supplied sterile, and should be steam autoclaved prior to first and each subsequent use (see Hygiene and Disposal section).

2. Place rubber dam or other suitable isolation technique.

3. Pre-operative Placement (optional)

1. WedgeGuard placement

Prior to preparation, select the appropriate sized WedgeGuard, and place interproximally. Palodent® V3 WedgeGuards contain a placement tab with hole, which is securely grasped by the pin tweezers. Do not use excessive force.

OR

2. Ring Placement

To effectively use a contoured sectional matrix, there must be enough interproximal space to position the matrix without force or distortion. The retaining ring, when placed prior to and left on during cavity preparation, provides the necessary separation with no additional chair time. **NOTE:** Do not place retaining ring pre-operatively if WedgeGuard is used. Follow these ring placement instructions after preparation completion, shield removal and matrix placement (step 7).

To place the ring, grasp it within at its widest diameter with the provided forceps, being sure the ring rests within the channel on the forceps beaks. Spread the forceps to expand ring only enough to clear tooth height of contour. Do not over-expand. Over-expansion may result in permanent ring deformation. The locking slide on the forceps may be engaged to facilitate carrying and placement of the ring.

Place the ring with one tine in each interproximal space from buccal to lingual adjacent to the surface to be restored. The direction the ring points, mesial or distal, does not matter, though placing it so that the ring points mesially is usually more easily achieved due to easier access.

With the tines properly positioned interproximally, release the locking slide, and slowly release the forceps tension. When ring is securely seated, release the forceps completely and withdraw forceps.

Small parts under tension.

- Be sure ring is securely seated in the forceps before introducing into the patient field.
- Be sure ring is securely seated apically before withdrawing forceps.

4. Cavity Preparation

Complete cavity preparation as usual. Once the preparation is complete, remove the ring or the metal shield of the WedgeGuard (if applicable). Use the pin tweezers to grasp the hole on the WedgeGuard shield, and slide shield laterally to remove. The wedge may remain in place if apical to the gingival cavosurface margin. Properly dispose of the shield with contaminated sharps.

5. Matrix Selection

The 5.5 mm Palodent® V3 contoured sectional matrix is recommended for most applications. The 4.5 mm and 3.5 mm are designed for use with adolescents and patients with poorly erupted posterior teeth. They are also ideal for the direct-indirect technique. The 6.5 mm matrix is designed for larger cavities occlusogingivally. It augments a contoured matrix with a gingival apron and an adjustable continuation of the contour at the marginal ridge. The 7.5 mm matrix is useful for patients with periodontal issues that require more coverage toward the gingival margin.

Technique Tip: Rings, Matrices, Wedges and WedgeGuards may be used with most popular matrix systems, including other sectionals. The Rings, Wedges and WedgeGuards may be used with most circumferential matrices. The benefits of the Ring, Wedge and WedgeGuard contours assist proper proximal contour.

6. Application of Matrix and Wedge

- Grasp the matrix with the pin tweezers using the hole located on the occlusal tab. If the ring was placed properly before tooth preparation, then the matrix should slide easily into place adjacent to the space to be restored from the occlusal approach. Check to be sure gingival tissue is not trapped between band and tooth surface. If so, slightly lift and reposition band. Ensure matrix is apical to preparation margin with no gap.

Technique Tip: If WedgeGuard wedge prevents matrix from fully seating gingivally, slide wedge out slightly, place matrix, then reposition wedge.

- Close the gingival margin by placing a contoured wedge or anatomic wooden wedge. Palodent® V3 wedges contain a placement tab with hole, which is securely grasped by the pin tweezers. This assures secure placement and minimizes the likelihood of dropping small accessories. **Technique Tip:** Palodent® V3 wedges' hollow design allows for easy placement of a second wedge from the opposing side (if needed) without dislodging the first. **Technique Tip:** Alternatively, the fork design allows placement of the Palodent® V3 retaining ring first if desired. The wedge can then be easily placed through the fork time without dislodging the ring.

7. Replace the Palodent® V3 retaining ring in the proximal plane, engaging the matrix and the wedge, thus stabilizing it against tooth structure. The forks of the tines should be sitting over the wedge.
8. Lightly burnish the matrix against the adjacent tooth both buccally and lingually to further adapt it to form proper contours. **NOTE:** Over burnishing may distort matrix and compromise contours. The pin tweezer has a built-in ball burnisher for this purpose. The matrix placement tab may be bent over the adjacent marginal ridge to facilitate restoration marginal ridge contour. **Technique Tip:** When restoring an MOD preparation, two rings may be placed facing opposite directions, one mesially and one distally or both pointing in the same direction, usually mesially, one over the other placing the mesial matrix and ring first.
9. **Placement of Restorative**
Place restorative material as per restorative material manufacturer's instructions for use and establish proximal contact as per usual technique. Light cure and/or allow material to reach initial set. Gross contouring may be accomplished with matrix in place.
10. **Matrix Removal**
Following appropriate restorative curing/setting, remove ring(s) using forceps, remove wedge(s) and sectional matrices using pin tweezers. Finish contouring and verify appropriate proximal contact. **Technique Tip:** It is advisable to "post-cure" light cured restorative materials following matrix removal from all surfaces, including proximo-facial and lingual line angles.

Small parts – To reduce risk of ingestion/inhalation.

Ensure matrix is grasped securely with pin tweezers, fingers or locking college pliers during withdrawal from the mouth.
Recover all pieces from the mouth immediately.

4. HYGIENE AND DISPOSAL

Cross-contamination – To reduce risk of Infection.

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Reprocess reusable products as described below.
- When not in use, all supplies and instruments should be in covered storage, such as drawers and cabinets, and away from potential contamination.
- During treatment clinicians with patient contact should only handle items to be used.

NOTICE

Wrong cleaning or disinfection method.

Damage to components.

- Ensure that phosphoric acid etchants are removed from the retaining ring tines prior to reprocessing.
- Most ultrasonic cleaning solutions will not affect the Palodent® V3 rings, although chemicals in some cleaners may cause tine discoloration. The discoloration is only cosmetic, but if that is a concern put the rings in a beaker with a water/detergent solution and place that in the ultrasonic cleaner to prevent contact with chemicals.

Pre-Use Cleaning – Matrices, Wedges, WedgeGuards.

Pre-use cleaning and disinfection may be omitted for single-use matrices, wedges and WedgeGuards stored in their original packaging and protected from contamination. Proceed to packaging and steam autoclaving according to instructions below.

4.1 Palodent® V3 instruments and components

Process		✓ Acceptable
Cleaning	Mechanical (Ultrasonic)	✓
	Manual	✓
	Automated (Washer/Disinfector)	✓
Disinfection	Automated (Washer/Disinfector)	✓
	Manual	✓
	Immersion	
Sterilization	Steam Autoclave	✓
	Immersion	
Single Use (Matrices, Wedges, WedgeGuards)	Dispose of after Use	✓
Reusable/Not Reprocessable	Dispose of if contaminated	

Instructions for cleaning, disinfecting and sterilizing Palodent® V3 Instruments and components	
Forceps, Pin Tweezers, Retaining Rings, Matrices, Wedges, WedgeGuards	
Warnings	• High-level disinfection has not been validated as a terminal process for the Palodent® V3 Instruments and components. Disinfection is not recommended as a terminal process. Proceed to Sterilization following cleaning and any optional/intermediate disinfection process. • Steam autoclaving sterilization is appropriate and recommended for the Palodent® V3 Instruments and components. • Follow proper infection prevention activities, such as proper hand washing and donning new puncture- and chemical-resistant gloves at pertinent steps. • Do not use hard wire brushes to clean the Palodent® V3 Instruments and components as scratching and mechanical degradation can occur. • Use only a disinfecting solution which is approved for its efficacy, EPA registered (and/or Health Canada approved) and use in accordance with the IFU of the disinfecting solution manufacturer. • Do not allow the Palodent® V3 Instruments and components to exceed 137 °C. • Do not use phenol-based glutaraldehyde. • Always use a pH neutral instrument cleaning solution if authorized. • Sterilize metal and plastic components in separate pouches to avoid damage to components. • If the integrity of the pouch has been compromised, the device should be processed again before use. • All devices should be cleaned and sterilized before each use. For the countries that require a three-step process, in which disinfection is required prior to sterilization, all devices must be cleaned, disinfected, and sterilized before use.

Limitations on Reprocessing	• Repeated reprocessing has minimum effect on these instruments. End of life is normally determined by wear and damage due to use. • Cold liquid immersion disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.
Initial Treatment at the Point of Use	• Do not allow residue or any form of contamination to dry on the device. Remove excess soil/gross debris by wiping, brushing, and/or rinsing under water. • Transport to processing area and follow validated processing instructions below. • It is recommended that the device be processed as soon as is reasonably practical following use.
Preparation before cleaning	Do not attempt to disassemble. Process the Palodent® V3 Instruments and components fully assembled.
Cleaning: Mechanical	• The chemical manufacturer's instructions for the cleaning and disinfection solution should be followed, observing concentration rates, contact times, fill level, degassing processes, etc. • The recommended solution is Resurge® Instrument Cleaning Solution mixed to 0.5 oz/gal (14.8 ml/3.8 l), at 35 °C-40 °C/95 °F-104 °F, FD 370 Cleaner, Dürr Dental AG, Germany or equivalent enzymatic cleaning solution. • Immerse Palodent® V3 Instruments and components in an ultrasonic bath containing the prepared solution for at least 20 minutes unless otherwise specified by the time specified by the solution manufacturer. • Rinse under tap water of at least drinking quality for a minimum of 30 seconds. • Allow to air dry. • Visually inspect for visible soil, and repeat the cleaning and disinfection procedure if necessary.
Cleaning and Disinfection: Manual	• Clean Palodent® V3 Instruments and components with a soft brush, tap water, and liquid soap until free of visible contamination. Pay special attention to device seams and insertions. • Remove cleaning solution residue with a cloth soaked with tap water or rinse under warm tap water for 2 minutes. • Dry Palodent® V3 Instruments and components with a lint-free, single-use cloth or allow to air dry in a clean area. • Visually inspect for visible soil, and repeat the cleaning procedure if necessary. Disinfection by Wiping • To disinfect Palodent® V3 Instruments and components, thoroughly wipe all surfaces, with a new single-use wipe in combination with an alcohol-based, tuberculocidal, quaternary ammonium cleaning and disinfecting solution. • Use additional wipes as needed to ensure the entire outer surface, including seams and crevices, remain visibly wet for the contact time recommended by the solution manufacturer. • Pay special attention to all seams and crevices while wiping. • Rinse the Palodent® V3 Instruments and components under tepid running water (40 °C, potable) for 30 seconds to remove disinfectant solution. • Dry the Palodent® V3 Instruments and components using a dry, lint-free disposable cloth or disposable wipe.
Cleaning and disinfection: Automated	• A thermal disinfector that complies with ISO 15883 may be used for automatic cleaning and disinfection. • Use only properly maintained, calibrated, and approved washer disinfectors according to ISO 15883. • Follow the manufacturer's instructions to select the solutions and programs. • Remove excess soil with a disposable cloth/paper wipe. • In case of gross contamination of Palodent® V3 Instruments and components, remove it using a soft brush and running water. • Place Palodent® V3 Instruments and components in the washer-disinfector allowing water and detergent to enter into and drain out of device openings. • Run washer-disinfection program with AO value ≥ 3000 (e.g. 5 min at ≥ 90°C) (e.g. Miele Vario TD) using appropriate detergents (e.g. Neodisher® MediClean and Neodisher® Z, both from Dr. Weigert, Germany or equivalent). • After the cycle is complete, visually inspect the Palodent® V3 Instruments and components for degradation due to extended use and reprocessing. If the device is discolored, cracked, worn, distorted, etc., discard and do not use. • Proceed to Sterilization following Automated Cleaning and Disinfection.
Drying	• The drying instructions are incorporated in the cleaning and disinfection section above.
Maintenance, Inspection and Testing	• Visually inspect the Palodent® V3 Instruments and components for degradation due to extended use and reprocessing. If the device is discolored, cracked, worn, distorted, etc., discard and do not use. • Forceps, tweezers, and other devices require no additional maintenance and should not be oiled. • Ensure moving parts operate smoothly. • If a Palodent® V3 retaining ring is deformed by over stretching during placement or removal, the ring should be discarded.
Packaging	• Ensure that the Palodent® V3 Instruments and components are completely dry before packaging for sterilization. • A lint-free, disposable cloth may be used to dry Palodent® V3 Instruments and components. • Place each Palodent® V3 device in a separate appropriately sized steam-sterilization pouch. • Use of FDA-cleared, ISO 11607 compliant Paper/plastic steam sterilization pouches (e.g., AssurePlus® Sterilization Pouches) is recommended.

Sterilization/ Steam autoclaving	- The Palodent® V3 Instruments and components must be sterilized utilizing steam autoclave sterilization. NOTE: Palodent® V3 Forceps, Pin Tweezers, Retaining Rings, Matrices, Wedges and WedgeGuards may be sterilized by Dynamic-Air-Removal/Pre-Vacuum steam autoclave. The following Pre-Vacuum cycles may be used for Forceps and Pin Tweezers: 134 °C (273 °F) for 3 minutes 30 seconds 135 °C (275 °F) for 3 minutes The following Pre-Vacuum cycles may be used for Retaining Rings, Matrices, Wedges and WedgeGuards: 132 °C (270 °F) for 4 minutes with a minimum drying time of 20 minutes 134 °C (273 °F) for 3 minutes with a minimum drying time of 18 minutes 135 °C (275 °F) for 3 minutes with a minimum drying time of 16 minutes NOTE: Palodent® V3 Forceps, Pin Tweezers, Retaining Rings, Matrices, Wedges and WedgeGuards may be sterilized by Gravity Displacement steam autoclave. The following Gravity cycles may be used for Forceps and Pin Tweezers: 135 °C (275 °F) for 10 minutes with a minimum drying time of 30 minutes The following Gravity cycles may be used for Retaining Rings, Matrices, Wedges and WedgeGuards: 132 °C (270 °F) for 15 minutes with a minimum drying time of 30 minutes 135 °C (275 °F) for 10 minutes with a minimum drying time of 30 minutes An alternate method of sterilization is to place non-bagged instruments into the steam sterilizer and run at one of the above listed cycles
Storage	- Do not remove from the pouch until ready for use to prevent contamination. - Instruments sterilized non-bagged should be used immediately. - Store at room temperature, away from moisture or excessive humidity. - To prevent contamination, store processed device in covered storage such as a drawer or cabinet until use. - Prior to use, inspect the pouch. If the integrity of the pouch has been compromised, the device should be processed again before use.
Additional Information	- Prior to use, inspect the device. Discard any Palodent® V3 Instrument or component which has become damaged, worn, or distorted. - The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for use/reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and trained personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process. Likewise, any deviation by the processor from the instructions provided should be properly evaluated for effectiveness and potential adverse consequences. Use of other reprocessing procedures/methods is at the discretion and sole responsibility of the practitioner.
Manufacturer Contact	Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative.

4.2 Disposal

- Dispose of in accordance with national and local regulations.
- During incineration, Wedges and WedgeGuards may produce irritating and highly toxic gasses.
- Used matrices are contaminated and sharp. When handling or disposing, use precautions as when handling or disposing of other contaminated sharps.

5. LOT NUMBER AND CORRESPONDENCE

- The following numbers should be quoted in all correspondence:
 - Reorder number
 - Lot number on the package
 - Copy of proof of purchase
- Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
- Device Identification (Basic UDI-DI): ++D002MATRIXSYSTEMSPX

© 2025 Dentsply Sirona Inc. All Rights Reserved. **559702vWEB** (R 1/9/25)

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