Prior to use, carefully read the instructions for use.

DIAPOLISHER PASTE

DIAMOND POLISHING PASTE FOR COMPOSITE AND PORCELAIN

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

Final high luster polishing of direct, indirect composite resin and porcelain restoration.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to benzoin preservatives.

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2) Hold the spraying upright and remove the wing cap by turning counter-clockwise.

3) Attach the dispensing tip correctly to the spraying by turning clockwise.

Note:
Take care not to screw the dispensing tip too tightly.
This may damage its screw. In order to ensure a tight connection make sure that the threads are free of debris.

4) After attaching the dispensing tip, protect it with the protective cap until ready for use in order to avoid contamination or drying-out.

II. Dispensing
Gently push the plunger and dispense sufficient material into a clean dispensing dish or on a mixing pad. The spraying can be stored either with the original cap or with the protective cap on the dispensing tip.

III. Polishing
Apply a small amount of paste onto an appropriate composite / porcelain polishing brush or cup. Polish all surfaces under moderate speed and pressure (up to 10,000 rpm). Refer to the respective manufacturer's instructions for use of the brush or cup. After polishing, rinse with water to remove the paste.

Note:
In case of contact with adjacent natural tooth structure, use moderate speed and pressure up to 5,000 rpm.

STORAGE

Recommended for optimal performance, store at room temperature away from heat, moisture and direct sunlight (4-25 °C) (39.2-77.0°F).

PACKAGES

Kit contains: DIAPOLISHER PASTE 2 g x 1.

Dispensing tip (needle type) x 1, Protective cap x 1

Refill: Dispensing tip (needle type) 20 pieces

CAUTION

1. To prevent splatter of paste and / or excessive heat generation always use rotation instruments at speeds below 10,000 rpm. Protect composite and porcelain, and below 5,000 rpm for natural tooth. Wear protective eye glasses during use.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. Wear gloves or rubber gloves during use to prevent possible sensitivity.

4. When polishing the polimert material, use a dust collector and wear a dust mask to avoid inhalation of dust particles.

5. The dispensing tip cannot be sterilized in an autoclave or chemilav.

6. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

7. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referred in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

or for The Americas:

<http://www.gcamerica.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTION

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them to the relevant authority or vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contact_en

as well as to our internal vigilance system:

vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Courts Counter Newport Pagell Buckinghamshire
MK16 8JS United Kingdom

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II. Dispensing

Gently push the plunger and dispense sufficient material into a clean dispensing dish or on a mixing pad. The spraying can be stored either with the original cap or with the protective cap on the dispensing tip.

III. Polishing

Apply a small amount of paste onto an appropriate composite / porcelain polishing brush or cup. Polish all surfaces under moderate speed and pressure (up to 10,000 rpm). Refer to the respective manufacturer's instructions for use of the brush or cup. After polishing, rinse with water to remove the paste.

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Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them to the relevant authority or vigilance system, by selecting the proper authority of your

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