

Tetric® PowerFill | Tetric® PowerFlow

1

Self-Etch



or Etch & Rinse



15-30 s
10-15 s



or

Selective Enamel-Etch



15-30 s



2

Adhese® Universal
VivaPen®



2-3 clicks



20 s



AIR



3

Optional

Tetric®
PowerFlow



up to
4 mm



4

Tetric®
PowerFill



up to
4 mm



Bluephase®
PowerCure



For permanent teeth only
For deciduous teeth see instructions for use

	Tetric PowerFill/ Tetric PowerFlow	Adhese Universal
500 – 900 mW/cm ²	20 s	10 s
1,000 – 1,300 mW/cm ²	10 s	10 s
1,800 – 2,200 mW/cm ²	5 s	5 s
2,700 – 3,300 mW/cm ²	3 s	3 s

Date information prepared: 2023-06-29
762056/Rev. 0

ivoclar

see
Instructions
www.ivoclar.com/eifu

Rx ONLY
For dental use only!


CE 0123

Tetric® Line

4-mm composites

SCULPTABLE
Tetric® PowerFill

FLOWABLE
Tetric® PowerFlow

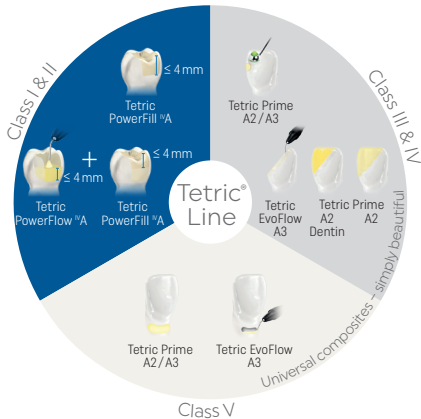

Conventional composites

SCULPTABLE
Tetric® Prime

FLOWABLE
Tetric EvoFlow®



Manufacturer: Ivoclar Vivadent AG
Bendererstrasse 2, 9494 Schaan/Liechtenstein

www.ivoclar.com

ivoclar

Just for patients with permanent teeth/ nur für Patienten mit bleibenden Zähnen